

Compat ella[®]

Compat Ella[®] Combiset 1.5L

The Compat Ella[®] Combiset 1.5L is intended for administration of enteral nutrition, fluids or medication and is suitable for use with the Compat Ella[®] feeding pump.

Features at a glance

The Compat Ella[®] Combiset 1.5L features an **integrated 1.5L Flexibaggle[®] reservoir**, an **ENFit[®] 3-way medication port** and an **ENFit[®] distal end** (patient side tube connection).

Built-in Safety: All Compat Ella[®] administration sets feature patented fail-safe **Auto-Anti-Free-Flow (AAFF)** technology. **AAFF** is incorporated into the cassette and is always automatically activated to ensure patient safety.


Safer Connectors: All Compat Ella[®] administration sets include ENFit[®] connectors made according to **ISO 80369-3**. The ENFit[®] connector is designed to be **IV Incompatible** to promote **patient safety** by helping to avoid accidental misconnections.

Convenience: The integrated 1.5L reservoir offers a convenient open system with a **built-in hanger and screw cap**. The **large top-fill opening** allows to refill the bag with nutrition or water without stopping the feeding program, or removing the reservoir from the hanger.

Food Safe Materials: All Compat Ella[®] administration sets are **DEHP, BPA and Latex-Free***. The Flexibaggle[®] 1.5L reservoir is made of Ethylene-vinyl Acetate (EVA), a soft and food friendly material.

Compatibility:

- Enteral feeding tubes: ENFit[®]
- Medication dispensers: ENFit[®]

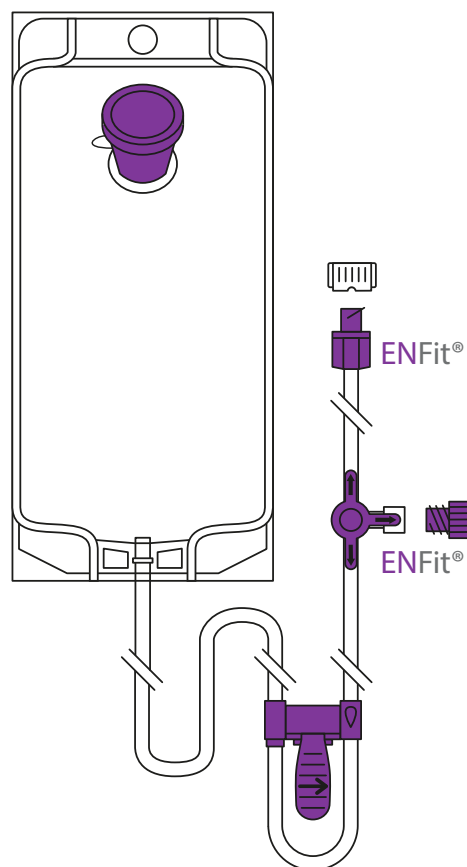


Ordering Information

Product Name	Product Code	Qty/Box
Compat Ella [®] Combiset 1.5L	12298202	30

Have you met Compat Ella[®]?

Compat Ella® Combiset 1.5L

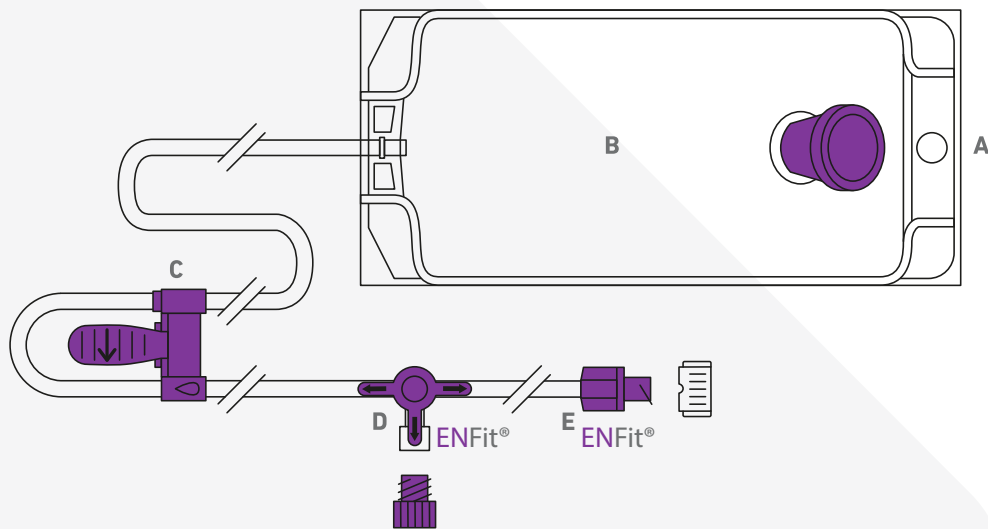

Components

- A** Large top-fill opening with built-in hanger and screw cap
- B** Flexibaggle® 1.5 L container made of EVA with graduations (at 50 mL and every 100 mL) and tags for patient name and other important information (formula, volume, date and rate)
- C** Compat Ella® pump cassette with integrated and patented Auto-Anti-Free-Flow (AAFF) safety feature
- D** ENFit® 3-way medication port with cap
- E** ENFit® distal end with cap (patient side tube connection)

Specifications

- Sterile:** Sterilized with Ethylene Oxide.
- Packing:** Individually packed in cases of 30 pieces (58.5 x 38.5 x 16 cm).
- Storage:** Store at room temperature. Keep away from sunlight and humidity.
- Shelf life:** 3 years from production date. Expiry date on the package.
- Duration of use:** 24 hours. Single Use.
- Tube Length:** Distance from bag to patient side tube connection: 230 cm. Distance from Flexibaggle® to pump: 50 cm.
- Tube diameters:** Nominal inner diameter: 3.0 mm. Nominal outer diameter: 4.15 mm.
- CE Mark:** Complies with the European Medical Device Directive 93/42 EEC. A CE Certification has been issued by an accredited Notified Body: TÜV SÜD (CE0123).
- Legal manufacturer:** Cedic Srl, Via Liberazione 63/9, IT-20068 Peschiera Borromeo (MI).
- Ecological Footprint:** The 'Grüne Punkt' symbol signals that we contribute to the financing of the recycling.
- Materials:** Tubing: PVC and silicone. All materials DEHP, BPA and Latex-free*. Additional details on request.
- Country of origin:** Made in Italy.

*Some incompatibilities may exist.
**BPA, Latex and Phthalates including DEHP are not intentionally added during the manufacturing process.
To obtain further information please contact your Compat® representative or go to www.compat.com
*Unless stated otherwise, all trademarks are used with permission.