Compat ella®


- DE Enterale Ernährungspumpe
- EN Enteral feeding pump
- ES Bomba de nutrición enteral
- FR Pompe de nutrition entérale
- IT Pompa di alimentazione enterale
- NL Enterale voedingspomp
- SV Enteral nutritionspump

Sprachverzeichnis / Language index / Índice de idiomas / Index des langues / Indice delle lingue / Taalindex / Språkindex

DE	Deutsch	Gebrauchsanweisung	Seite	05
EN	English	Instructions for Use	Page	38
ES	Español	Instrucciones de uso	Página	68
FR	Français	Mode d'emploi	Page	98
IT	Italiano	Istruzioni per l'uso	Pagina	128
NL	Nederlands	Gebruiksaanwijzing	Pagina	158
SV	Svenska	Bruksanvisning	Sida	188

Einlegen der Kassette / Loading the Cassette / Carga del cassette / Chargement de la cassette / Caricamento del set / Cassette laden / Ladda kassetten

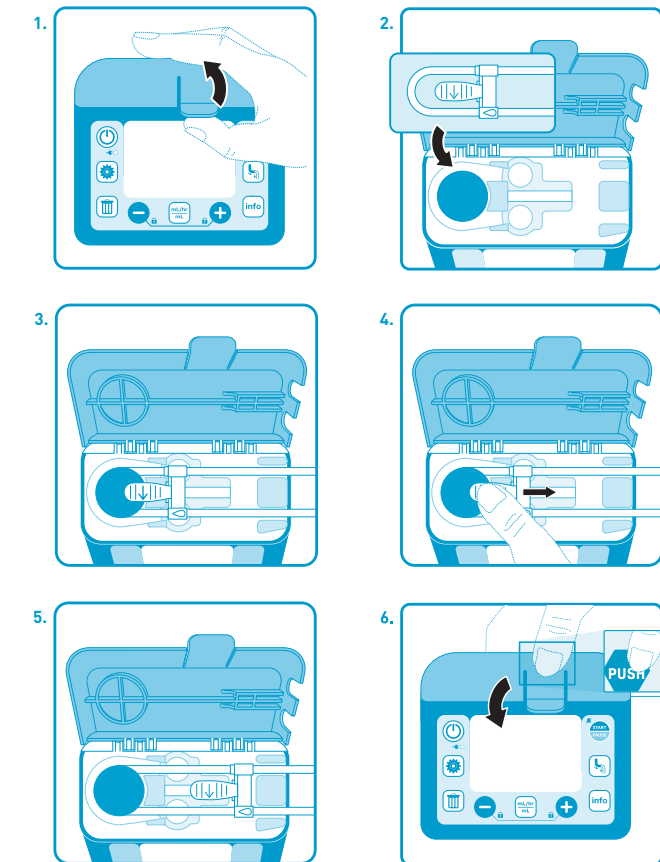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Instructions for Use for Compat Ella® Pump

Table of Contents

1. Introduction	39	8. Feeding in Continuous Mode	51
1.1 Intended Use	39	9. Feeding in Interval Mode	52
1.2 Support	39	10. Pausing a Nutritional Therapy	54
2. Safety Information	39	11. Delivery Information	55
2.1 Warnings and Precautions	39	11.1 View/Clear Delivery Information in Continuous Mode	55
2.2 Contraindications	40	11.2 Rolling 30-day Volume History	55
2.3 General complications associated with enteral tube feeding	41	11.3 View/Clear Delivery Information in Interval Mode	56
2.4 Specific precautions related to the compatible sets and nutrition	41	12. User Settings	57
3. System Overview	42	12.1 Alarm level, Backlight, Autodim, End of Nutritional Therapy Alarm	57
3.1 Pump (see pages 3-4)	42	12.2 Changing the Language	58
3.2 Administration Set (see page 4)	42	13. Locking and Unlocking the Keypad	58
3.3 Button Function	42	14. Advanced Settings	59
3.4 Display	43	15. Alarms & Troubleshooting	60
3.5 Battery	44	15.1 Alarm Handling	60
4. Pump Mode Overview (Continuous vs. Interval)	45	15.2 Alarm Overview and Troubleshooting	61
4.1 Continuous Mode: Menu Overview	46	16. Cleaning & Disinfection	63
4.2 Interval Mode: Menu Overview	46	17. Maintenance	64
5. Clear Nutritional Therapy Settings and Delivery Information	47	18. Accessories	64
6. Pump Setup	47	19. Limited Warranty	65
6.1 Connect to the AC power adapter	47	20. List of Symbols	65
6.2 Use with a pole clamp	47	21. Specifications	66
6.3 Power On and Off	48	22. Recycling and Disposal	67
6.4 Inserting the Administration Set	48		
6.5 Filling the Administration Set (Priming)	49		
7. Quick Start Guides	50		
7.1 Continuous Mode Feeding	50		
7.2 Interval Mode Feeding	50		

1. Introduction

Compat Ella® is a small, lightweight and portable rotary peristaltic pump used to deliver programmed amounts of enteral tube feed at selectable delivery rates to the gastrointestinal system. The pump should be used only after proper training regarding care, use and maintenance of the Compat Ella® pump and accessories.

1.1 Intended Use

The Compat Ella® enteral feeding pump is intended to be used for the delivery of enteral tube feed and hydration at controlled rates to the gastrointestinal system. It is intended for use with adult and pediatric patients requiring enteral feeding and hydration in home and professional healthcare environments.

1.2 Support

If you need help with the Compat Ella® pump, please contact your sales representative or go to www.compat.com.

2. Safety Information

Read the entire Manual before operating the Compat Ella® pump.

Read the entire administration set IFU (Instructions for Use) before installing an administration set in the Compat Ella® pump.

Warning: Describes a hazard or unsafe practice that if not avoided can result in severe bodily injury, death or property damage.

Precaution: Describes a hazard or unsafe practice that if not avoided can result in minor bodily injury or mainly property damage.

Note: Describes something that is more important than the rest of the text.

2.1 Warnings and Precautions

Warnings:

- Not for intravenous use. For enteral use only.
- If pump is damaged or altered, do not use it.
- Make sure power cord plug is completely dry before and during use.
- Do not attempt to attach the “country specific” AC adapter to AC power without it attached to the AC adapter body.
- Do not use in the presence of flammable anesthetics or other flammable gases (including enriched oxygen environments), near flammable fluids or objects, or in the presence of oxidizing agents.
- During pump start up, confirm all the screen segments are visible. If any are missing or if the screen is not readable, do not use the pump.
- When turning the pump on, if you do not hear the tones, do not use the pump.
- Use only prescribed enteral tube feed supplied by a qualified healthcare professional and hydration fluids such as water, tea.
- Use only Compat Ella® administration sets.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of the pump could result in increased electromagnetic emissions or decreased electromagnetic immunity of the pump and result in improper operation.
- Inspect the Compat Ella® administration set. Do not use if damaged or altered.

- Always ensure appropriate, timely and regular patient monitoring, especially if an interruption of the tube feeding delivery may lead to a patient hazard.
- Replace the Compat Ella® administration set every 24 hours to maintain delivery accuracy and help prevent the growth of potential harmful contamination.
- Give particular attention to the risk of strangulation with cables and sets.
- Do not program Rate and Volume combinations that exceed 24 hours.
- Do not submerge the pump in water.
- Do not use in a dishwasher.
- To avoid electrical shock, never clean while the pump is on or plugged in.
- Do not autoclave.
- The use of harsh cleaners/disinfectants and isopropyl alcohol may damage the pump housing.
- Do not disassemble the pump, remove the labels or modify the device.

Precautions:

- Do not use in strong magnetic fields. e.g. MRI (Magnetic Resonance Imaging) environments.
- Use of the Compat Ella® pump adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, the pump and the other equipment should be observed to verify that they are operating normally.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Compat Ella® pump, including cables specified by the manufacturer. Otherwise, degradation of the performance of the pump could result.
- Ensure that the administration set is not connected to a patient when priming.
- Verify that the administration set is correctly installed, is connected to the nutrition container and is ready to prime.
- When programming a new therapy, make sure to clear the **VOLUME GIVEN THIS THERAPY** parameter.
- After starting a therapy, verify all therapy parameters (including rate, dose, and volume given this therapy).
- The battery capacity is an estimate. If you are unsure that enough capacity remains for your intended use, connect to the AC power until recharged.
- The advanced settings should only be adjusted by a qualified healthcare professional.

2.2 Contraindications

The Compat Ella® Pump should not be used:

- If enteral feeding is contraindicated;
- For the administration of intravenous infusions.
- With sets other than “Compat Ella®” administration sets
- If intestinal obstruction
- If ileus (Inability of the intestine to contract normally/ move waste out of the body)
- If high output small bowel fistula
- If high doses positive inotropic agents
- Enteral feeding can be given in the presence of gut obstruction or when there is a high output small bowel enterocutaneous fistula
- If enterocutaneous fistula

2.3 General complications associated with enteral tube feeding

Mechanical complications

- Tube obstruction
- Primary malposition
- Perforation of the intestinal tract
- Secondary displacement of the feeding tube
- Knotting of the tube
- Accidental tube removal
- Breakage and leakage of the tube Leakage and bleeding from insertion site
- Erosion, ulceration and necrosis of skin and mucosa
- Intestinal obstruction (ileus)
- Hemorrhage
- Inadvertent IV infusion of enteral diet

Infectious complications

- Infection at the tube insertion site
- Aspiration pneumonia
- Nasopharyngeal and ear infections
- Peritonitis
- Infective diarrhea

Metabolic complications

- Electrolyte disturbances
- Hyper- and hypoglycemia
- Vitamin and trace element deficiency
- Tube feeding syndrome (“Refeeding syndrome”)

2.4 Specific precautions related to the compatible sets and nutrition

Due to the diversity of nutritional solutions used, which have a wide range of characteristics (i.e. density, viscosity, quantity of dissolved gasses, rheology, particles, temperature, individualized nutrition preparation), in certain circumstances the performance of the pump may be limited, due to occlusion of the feeding set or tube. In all circumstances such an incident will be detected by the Compat Ella® pump and will trigger an occlusion alarm.

We recommend only using the Compat Ella® pump with timely and regular patient monitoring, especially in sensitive patient population where an interruption of the nutrition delivery may lead to a patient hazard.

Therefore, when Compat Ella® pumps are used to feed patients who require a constant supply of nutrition (for whom the disruption of food supply can lead to critical situations), it is necessary to only use commercially available enteral nutritional solutions prepared as described by the manufacturer, use of compatible sets and appropriate and regular monitoring food administration.

3. System Overview

3.1 Pump (see pages 3-4)

- | | | |
|--------------------------------|-------------------------------------|---|
| A Pump door | G Upstream pressure sensor | M Rear pump label with Quick Start Guide |
| B Door latch | H Downstream pressure sensor | N Loudspeaker |
| C Pump rotor | I Air sensor | O Serial number label |
| D Keypad | J Rubber foot pad | P Legal manufacturer label |
| E AC power status light | K AC power connection | Q AC power adapter |
| F Colour display | L Pole clamp fitting | R Pole clamp |

3.2 Administration Set (see page 4)

- | | |
|---------------------------|---------------------------------|
| S ENPlus connector | U ENFit® Medication port |
| T Cassette | V ENFit® Distal end |

Note: Different versions of the administration sets are available. To obtain further information please contact your sales representative or go to www.compat.com.

3.3 Button Function

The pump always retains in memory the on-screen settings and values when scrolling from one screen to the next, as well as when the pump is turned off.

Button	Primary Function	Secondary Function
	Turn pump on or off (hold 2 seconds [2"])	
	Start / Pause pump	Snooze alarm
	Auto-prime (hold 2 seconds [2"]) Press again to stop auto-prime early	
	Scroll through rate, volume and interval programming	Switch between continuous and Interval modes (hold 5 seconds [5"])
	Decrease and increase values (hold: accelerates speed)	Lock and unlock keypad (press both buttons together)

	Clear the on-screen value to default or 0	Reset all program values to default and clear recent feed history (hold button 3 seconds (3"))
	Scroll through recent feed history	Access last rolling 30-day Volume History (hold button 2 seconds (2"))
	Scroll through pump settings	Change language (See section 12.2)

 **Precaution:** Ensure that the administration set is not connected to a patient when priming.


3.4 Display

 **Warning:** During pump start up, confirm all the screen segments shown in the diagram hereafter are visible. If any are missing or if the screen is not readable, do not use the pump.

Screen Layout

The Compat Ella® pump features a colour display with alphanumeric characters, symbols and a backlight.

- 1 Battery charge level
- 2 Plugged into AC power
- 3 Keypad locked
- 4 Interval mode activated
- 5 Tube feed container empty / Air In Line
- 6 Alarm indicator
- 7 Run icon with occlusion indicators: the arrow segments light sequentially when the pump is running.
- 8 Main info line: displays alpha numerical values for programming and alarms.
- 9 Main info label: mL, h, or mL/h.
- 10 Text message line: provides miscellaneous info on pump status, programming and alarms.

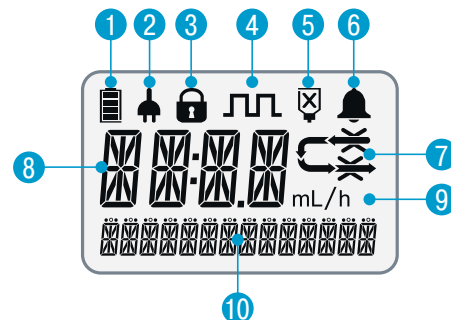Note: The backlight of the display turns off 10 seconds after the last key is pressed unless the pump is plugged into the AC power and the Autodim feature is deactivated.

Colour Screen Overview

The pump screen uses different colours to help distinguish pump status and on-screen information.

Programming information:
black characters


Normal operating conditions:
green icons with **black** characters

Alarms:
red characters and icons

Information:
blue characters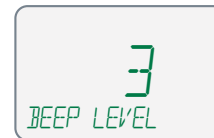


Settings:
green characters

Note: When the backlight is dimmed, all on-screen information is shown in black.

Default / Run Screen Overview

When the pump is running normally, the following screen will be shown:

1. The sequentially-appearing green arrows  show that the pump is currently running a programmed nutritional therapy.
2. The 120 mL/h shows the rate the pump is running in mL/h.
3. The 100/500 mL shows that 100 mL has been given out of the 500 mL programmed volume. If /CONT is displayed, this means no volume limit has been set and the pump will run continuously until the tube feed runs out.
4. The battery is fully charged and is plugged into the AC power.

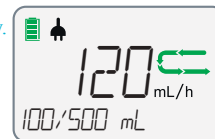Note: Compat Ella® only pumps fluid a little at a time. Normal function is to pump fluid for a while, then stop pumping fluid and then start pumping fluid again. This means that you will not hear the motor running all the time. The sequentially-appearing green arrows  are continually shown, which means that the pump is functioning correctly.

3.5 Battery

- First Use: Plug in the pump and fully charge the battery for a minimum of 6 hours before using the pump on battery power only. This will maximize the charge and life of the battery.
- Battery capacity: 24 hours at 125 mL/h.
- Recharge: approximately 6 hours from fully depleted to fully charged.

Battery Charge Indicators

Battery Fully Charged

Battery Partially Charged

Pump on External Power

Battery Charging

Precaution: The battery capacity is an estimate. If you are unsure that enough capacity remains for your intended use, connect to the AC power until recharged.

4. Pump Mode Overview (Continuous vs. Interval)

The Compat Ella® portable enteral feeding pump has two operational modes: Continuous and Interval.


Continuous Mode

Continuous mode is used to deliver tube feed at defined rates, with or without a volume limit.

Interval Mode

Interval mode allows a defined bolus to be delivered multiple times during the day at programmed intervals, with or without a total volume limit. An Interval is the time from the start of one bolus to the start of the next bolus.

Switch Between Continuous and Interval Modes

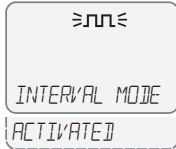Button	Direction / Result
	To change between operating modes, press and hold  for 5 seconds (5"). A specific icon  shows that Interval mode is activated. A message will appear confirming activation or de-activation. 

Note: A qualified healthcare professional must enable the use of the interval mode. (See advanced settings)

4.1 Continuous Mode: Menu Overview

Press  to scroll through **SET RATE** and **SET VOLUME**.
Pressing  again cycles back to **SET RATE**

Button	Device Display	Function
	SET RATE	The delivery speed in mL/h at which the pump will run. • Rate may be set from 1-600 mL/h in 1 mL/h increments. • Default value 0 mL/h
	SET VOLUME	The volume of tube feed to be given, in mL. • Volume may be set from 1-4000 mL in 1 mL increments. • Default value: CONT *
	SET RATE	Return to SET RATE screen

Note:

- Both **RATE** and **VOLUME** must be set / reviewed if either one is changed or the pump has just been turned on. Only then can the pump be started.

* Continuous: no feeding volume has been set. The pump will run continuously until the formula runs out.

4.2 Interval Mode: Menu Overview

Press  to scroll through **SET RATE** and **SET BOLUS**, **SET INTERVAL** and **SET TOTAL VOLUME**. Pressing  again cycles back to **SET RATE**

Button	Device Display	Function
	SET RATE	The delivery speed in mL/h at which the pump will run. • Rate may be set from 1-600 mL/h in 1 mL/h increments. • Default: 0 mL/h
	SET BOLUS	The bolus volume of tube feed to be given before each interval pause, in mL. • Bolus volumes may be set from 1-2000 mL in 1mL increments. • Default: 0 mL
	SET INTERVAL	The interval between start of one bolus and start of the next bolus. • Interval may be set from 00:15-12:00 h in 15-minute increments. The minimum interval possible may be greater than 00:15 depending on rate and bolus values entered in previous steps. • Default: 00:15 h
	SET TOTAL VOLUME	The total volume in mL to be given during the entire nutritional therapy. • Total volume may be set from 1-4000 mL in 1mL increments. • Default value: CONT *
	SET RATE	Return to SET RATE screen

Note: RATE, BOLUS, INTERVAL and TOTAL VOLUME must all be set / reviewed if either one is changed or the pump has just been turned on. Only then can the nutritional therapy be started.

5. Clear Nutritional Therapy Settings and Delivery Information

Button	Direction / Result
	Hold down  for 3 seconds (3") to clear all nutritional therapy settings. • Nutritional therapy values cleared to default: <i>RATE, VOLUME, BOLUS, INTERVAL, TOTAL VOLUME</i> • Volume history values cleared to default: <i>VOLUME GIVEN THIS THERAPY, CUMULATIVE VOLUME</i> • The pump displays a <i>THERAPY RESET</i> confirmation message.

Note: This does not affect any other pump settings (such as alarm level, backlight etc.), language or rolling 30-day volume history.

6. Pump Setup

6.1 Connect to the AC power adapter

See pump and accessories overview (page 4)

The AC power adapter is delivered with multiple country-specific adapters. To change the country specific adapter, press the button on the AC power adapter and rotate country specific adapter to release it. Install the new country specific adapter and rotate until you hear a click.

Warning: Do not attempt to attach the “country specific” AC adapter to AC power without it attached to the AC power adapter body.

6.2 Use with a pole clamp

See pump and accessories overview (page 4)

6.3 Power On and Off

Button	Direction/Result
	Power On Press and hold  for 2 seconds (2") to power on. You should hear 3 tones. The pump will first display all screen segments briefly. Ensure that all the segments are visible. The pump will then perform a diagnostic and, if successful, will display <i>SELF CHECK OK</i> .
	Power Off Press and hold  2 seconds (2") to power off. You should hear 3 tones as the pump shuts down. <i>Note: All nutritional therapy parameters, volume history and user settings are retained in the pump's memory for re-use when the pump is next turned on.</i>

 **Warning:** When turning the pump on, if you do not hear the tones or do not see all the screen segments, do not use the pump. Contact your sales representative or an approved service provider.

Power Status Light

Solid green = Pump is plugged into the AC power.

6.4 Inserting the Administration Set

-  **Warnings:**
- Use only Compat Ella® administration sets.
 - Inspect the Compat Ella® administration set. Do not use if damaged or altered.
 - Replace the Compat Ella® administration set every 24 hours to maintain delivery accuracy and help prevent the growth of potential harmful contamination.

Loading the Cassette

See sequence on page 2 to load set into the pump.

Fig. 1: open door by lifting the thumb tab

Fig. 2, 3: wrap cassette loop around rotor. Note the arrow which shows correct orientation

Fig. 4, 5: using your thumb, slide the cassette to the right to clip the cassette in the pump

Fig. 6: close the door by pressing on the "PUSH" symbol.

Note: Compat Ella® Administration Sets are equipped with a patented Auto-Anti-Free-Flow (AAFF) mechanism, which is always active when needed. The AAFF mechanism is only released when the cassette is inserted in the pump and the door is closed.

6.5 Filling the Administration Set (Priming)

Priming fills the administration set with tube feed.

Precautions:

- Before priming, always verify that the administration set is not connected to the patient.
- Verify that the administration set is correctly installed, is connected to the nutrition container and is ready to prime.

Auto-prime

Auto-prime should always be used if possible.

Button	Direction/Result
	Press and hold  for 2 seconds (2'') to prime automatically. While priming, the pump displays the run icon together with a PRIMING message. You may stop auto-prime at any time by pressing  again.

Notes:


- The pump is programmed to automatically deliver a pre-set volume of tube feeding to prime the tubing.
- Once the pre-set volume of tube feeding has been given, the pump stops automatically and displays **END OF PRIMING**.
- When prime is complete, the tubing should be checked to ensure that the tubing is completely filled. If additional priming is required, the auto-prime can be repeated.
- The air detection alarm is disabled during priming.

Manual Prime

If the auto-prime function is unwanted, manual priming can be performed. Gently pinch the tubing on the cassette as shown and hold this position until all air is removed from tubing. If needed, gently squeeze and tilt container at the same time to assist fluid flow. If fluid does not flow, the pinch pressure may be too strong.

Note:

- It is important to only gently pinch the tubing above the symbol  to avoid damaging the Auto-Anti-Free-Flow mechanism located underneath. Use only the flat, soft part of your finger. Do not use fingernails as this may puncture the tubing.
- High viscosity feeds may not flow effectively using the manual priming method and should be primed using the pump auto-prime function

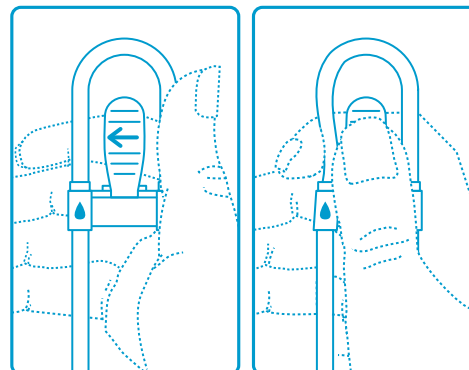7. Quick Start Guides

7.1 Continuous Mode Feeding

Step	Button	Function
1	 2"	Turn the pump on
2		Review the <i>VOLUME GIVEN THIS THERAPY</i>
3		Clear the <i>VOLUME GIVEN THIS THERAPY</i>
4	 2"	Prime the administration set
5		Use  and  to <i>SET RATE</i> 1 – 600 mL/h
6		Use  and  to <i>SET VOLUME</i> 1 – 4000 mL/CONT*
7		Start the nutritional therapy

* Continuous: no feeding volume has been set. The pump will run continuously until the formula runs out.

Precautions:

- When programming a new therapy, make sure to clear the *VOLUME GIVEN THIS THERAPY* parameter.
- After starting a therapy, verify all therapy parameters (including rate, dose, and volume given this therapy).

7.2 Interval Mode Feeding

Step	Button	Function
1	 2"	Turn the pump on
2		Review the <i>VOLUME GIVEN THIS THERAPY</i>
3		Clear the <i>VOLUME GIVEN THIS THERAPY</i>
4	 2"	Prime the administration set
5		Use  and  to <i>SET RATE</i> 1 – 600 mL/h
6		Use  and  to <i>SET BOLUS</i> 1 – 2000 mL
7		Use  and  to <i>SET INTERVAL</i> 0:15 h – 12:00 h
8		Use  and  to <i>SET TOTAL VOLUME</i> to be given 1 – 4000 mL/CONT*
9		Start the nutritional therapy

8. Feeding in Continuous Mode

Continuous mode is used to deliver tube feed at defined rates, with or without a volume limit.

Step	Button	Function	Instruction
1	 2"	Turn the pump on	The most recent nutritional therapy settings are always saved in memory. When power is turned on, the display shows the last programmed flow rate and volume and the actual volume given.
Load the administration set in the pump			
2		Review the volume given this therapy	Press  and review the VOLUME GIVEN THIS THERAPY .
3		Clear the volume given this therapy	If desired, press  to clear the VOLUME GIVEN THIS THERAPY . <i>Note: This should always be done when starting a new nutritional therapy.</i>
4	 2"	Prime the administration set	• Hold 2 seconds (2") to activate auto-prime. The pump will fill the administration set with a predetermined volume. • Press again to stop priming at any time. <i>Note: Do not prime when connected to the patient's feeding tube.</i>
5		Set the rate	Press  to access the SET RATE screen and use  and  to define the rate of tube feed delivery. • The rate may be set from 1-600 mL/h in 1 mL/h increments. <i>Note: Press  to reset value.</i>
6		Set the volume	Press  to scroll to the SET VOLUME screen and use  and  to define the amount of tube feed to be delivered. • Feeding with a dose limit: The value may be set from 1-4000 mL in 1 mL increments. • Continuous feeding without volume limit: Set the value to CONT (Continuous), the pump will run continuously until the nutrition container is empty. <i>Note: Press  to reset value.</i>

Ensure nutritional therapy settings correspond to the nutritional therapy prescribed.

Connect the administration set to the patient's feeding tube.

7		Start the nutritional therapy	Press  to run the nutritional therapy. The pump will automatically run the nutritional therapy and display the progress.
---	---	-------------------------------	---

Precautions:

- When programming a new therapy, make sure to clear the **VOLUME GIVEN THIS THERAPY** parameter.
- After starting a therapy, verify all therapy parameters (including rate, dose, and volume given this therapy).

Notes:

- Holding  or  changes values rapidly:
 - When setting rate, the value will pause at 1, 50, 125, 250, 600 mL/h.
 - Total Volume will also pause at 1, 125, 250, 500, 750, 1000, 1250, 1500, 2000 and 4000 mL.
- When at maximum value, pressing  again will go to the minimum value and vice versa e.g. rate from 600 mL/h to 1 mL/h.
- The pump will sound the alarm when a programmed volume is completely given or the nutrition container runs out. The **END OF THERAPY** alarm may be temporarily deactivated (see 12.).
- Patients sensitive to nutrition delivery should be monitored closely at all times.

9. Feeding in Interval Mode

Interval mode is used to deliver multiple bolus volumes at regular intervals, at defined rates, with or without a volume limit.

Step	Button	Function	Instruction
1	 2"	Turn the pump on	The most recent nutritional therapy settings are always saved in memory. When power is turned on, the display shows the last programmed flow rate and volume and the actual volume given.
Load the administration set in the pump			
2		Review the volume given this therapy	Press  and review the VOLUME GIVEN THIS THERAPY .
3		Clear the volume given this therapy	If desired, press  to clear the VOLUME GIVEN THIS THERAPY . <i>Note: This should always be done when starting a new nutritional therapy.</i>

4	 2"	Prime the administration set	• Hold 2 seconds (2") to activate auto-prime. The pump will fill the administration set with a predetermined volume. • Press again to stop priming at any time. <i>Note: Do not prime when connected to the patient's feeding tube.</i>
5		Set the rate	Press  to access the SET RATE screen and use  and  to define the rate of tube feed delivery. • Rate may be set from 1-600 mL/h in 1 mL/h increments. <i>Note: Press  to reset value.</i>
6		Set the bolus volume	Press  to scroll the SET BOLUS screen and use  and  to define the bolus volume or the amount of tube feed to be given multiple times. • Bolus may be set from 1-2000 mL in 1 mL increments <i>Note: Press  to reset value.</i>
7		Set the interval	Press  to scroll the SET INTERVAL screen and use  and  to define the time from the start of one bolus to the start of the next bolus. • Interval may be set from 00:15-12:00 h in 15-minutes increments. • The minimum interval possible may be greater than 00:15 depending on rate and bolus values entered in previous steps. <i>Note: Press  to reset value.</i>
8		Set the total volume	Press  to scroll the SET TOT VOLUME screen and use  and  to define the total amount of tube feed to be given if desired. • Feeding with a Dose Limit: The value may be set from 1 bolus programmed in step 6 to 4000 mL in 1 mL increments. The pump will run multiple bolus until the Total Volume limit has been reached. • Feeding without Dose Limit: Set the value to CONT (CONTINUOUS), the pump will run multiple bolus until the nutrition container is empty. <i>Note: Press  to reset value.</i>

Ensure nutritional therapy settings correspond to the nutritional therapy prescribed.

Connect the administration set to the patient's feeding tube.

9		Start the Nutritional Therapy	Press  to run the nutritional therapy. The pump will automatically run the nutritional therapy and display the progress.
---	--	-------------------------------	--

Precautions:

- When programming a new therapy, make sure to clear the **VOLUME GIVEN THIS THERAPY** parameter.
- After starting a therapy, verify all therapy parameters (including rate, dose, and volume given this therapy).

Notes:

- Holding **+** or **-** changes values rapidly:
 - When setting rate, the value will pause at 1, 50, 125, 250, 600 mL/h.
 - When setting Bolus, the value will pause at 1, 125, 250, 500, 750, 1000, 1250 and 1500 mL.
 - Total Volume will also pause at 1, 125, 250, 500, 750, 1000, 1250, 1500, 2000 and 4000 mL.
- When at maximum value, pressing **+** again will go to the minimum value and vice versa e.g. rate from 600 mL/h to 1 mL/h.
- The pump will sound the alarm when a programmed volume is completely given or the nutrition container runs out. The **END OF THERAPY** alarm may be temporarily deactivated (see 12.).
- After each bolus is complete, the pump will display an **INTERVAL PAUSE** and **TIME TO NEXT BOLUS** screen showing when the next bolus will start. This cycle will continue until the pump is turned off, a defined amount of tube feed is given or the tube feed runs out.

10. Pausing a Nutritional Therapy

Button	Result
	At any time while the pump is running, you may pause the nutritional therapy by pressing . The Run icon disappears and the pump displays a PUMP PAUSED message. Press  to resume the nutritional therapy. Use the Pause mode to: • Change the feeding program • Change the feed container • Administer medication without switching the pump off • Silence an alarm and correct problems

Note: After 2 minutes, an alert sounds and the message **PUMP WAITING** appears in the display. Press  to snooze the alarm and to extend the pause mode by an additional 2 minutes.

11. Delivery Information

11.1 View/Clear Delivery Information in Continuous Mode

At any time, press  to review the delivery information.

To clear : Ensure the pump is paused. Press  and then  to clear value to 0.

Button	Device Display	Function
  	VOLUME GIVEN THIS THERAPY	Display the volume in mL given for the current nutritional therapy. <i>Note: Must be manually cleared at the beginning of each new nutritional therapy. If not cleared before each new nutritional therapy, the end of therapy alarm will sound prematurely if a VOLUME has been set.</i> <i>To clear: Ensure the pump is paused. Press  and then  to clear value to 0.</i>
 	CUMULATIVE VOLUME	The cumulative volume given by the pump since the value was last cleared. Can be used to track volume history during any defined period, such as a shift or multiple nutritional therapies. <i>To clear: Ensure the pump is paused. Press  twice and then  to clear value to 0.</i>
		Exit to main screen.

*Note: if the volume information exceeds 9999 mL, the pump continues to display 9999 mL. Clear value to restart counting from 0.
If no button is pressed within 10 seconds, the info screens are exited.*

11.2 Rolling 30-day Volume History

Button	Function
   2"	Hold  for 2 seconds (2") to access the rolling 30-day volume history. Use  /  to scroll backwards and forwards by date. The value shown is the total volume given per 24-hour period, midnight to midnight for the last 30 days.

Note: The rolling 30-day volume history cannot be cleared.

11.3 View/Clear Delivery Information in Interval Mode

At any time, press  to review the delivery information and programmed nutritional therapy details. Press  to scroll through each menu item below. When the last menu is reached, pressing the  button exits the info mode. To clear **VOLUME GIVEN THIS THERAPY** or **CUMULATIVE VOLUME**: Ensure the pump is paused. Press  and then  to clear value to 0.

Button	Device Display	Function
	VOLUME GIVEN THIS THERAPY	Display the volume in mL given for the current nutritional therapy. <i>Note: Must be manually cleared at the beginning of each new nutritional therapy. If not cleared before each new nutritional therapy, the end of therapy alarm will sound prematurely if a VOLUME has been set.</i> <i>To clear: Ensure the pump is paused. Press  and then  to clear value to 0.</i>
	CUMULATIVE VOLUME	The cumulative volume given by the pump since the value was last cleared. Can be used to track volume history during any defined period, such as a shift or multiple nutritional therapies. <i>To clear: Ensure the pump is paused. Press  twice and then  to clear value to 0.</i>
	TIME TO THERAPY END	The remaining time (in hours and minutes) until the end of the nutritional therapy. <i>Note: Not available if CONT has been set for TOTAL VOLUME.</i>
	TIME TO BOLUS END OR TIME TO NEXT BOLUS	- When the bolus is being given, the time left in hours and minutes before the pump finishes delivering the bolus volume and enters interval pause. - When the pump is paused between bolus, the amount of time left in hours and minutes until the pump will start delivering the next bolus.
	BOLUS SET	The bolus volume in mL that has been programmed.
	INTERVAL SET	The interval that has been programmed in hours and minutes. An interval is the time from the start of one bolus to the start of the next bolus.
		Exit to the main screen.

Note: If no button is pressed within 10 seconds, the info screens are exited.

12. User Settings

12.1 Alarm level, Backlight, Autodim, End of Nutritional Therapy Alarm

At any time, press  to review and / or adjust the pump settings. Press  again to scroll between *BEEP LEVEL*, *BACKLIGHT*, *AUTODIM?* and *ALARM AT END OF THERAPY?*. When the last setting is reached, pressing  exits the option setting mode.

Use  and  to adjust any given setting. A setting is saved in memory until it is changed. Press  to set the on-screen value to default.

Button	Device Display	Function
	<i>BEEP LEVEL</i>	Set the audio alarm volume level from 1-3 (3 = highest). Default=3.
	<i>BACKLIGHT</i>	Set the luminosity of the screen backlight from 1-3 (3 = highest). Default=2.
	<i>AUTODIM?</i>	Set whether the screen backlight automatically dims when pump is connected to the AC power. <i>YES:</i> Screen dims after a 10-second delay. <i>NO:</i> Screen stays on permanently (not available when running on battery power) (Default). <i>Note: When the backlight is dimmed, all on screen information is shown in black.</i>
	<i>ALARM AT END OF THERAPY?</i>	Set whether the pump will warn the user the nutritional therapy is finished or not. <i>YES:</i> Pump will warn user (Default). <i>NO:</i> Pump will remain silent at end of nutritional therapy.
		Exit to main screen

Note: If no button is pressed within 10 seconds, the settings screens are exited.

12.2 Changing the Language

Step	Button	Direction
1		Press and hold  for 2 seconds (2") to activate language selection.
2		Use   to scroll to the desired language.
3		Press and hold  again for 2 seconds (2") to select the language.
4		Immediately turn the pump off to save chosen language. Restart the pump. <i>Note: If the pump is not immediately turned off, the language change will not be saved.</i>

13. Locking and Unlocking the Keypad

Use the keypad lock to avoid unintentional button presses, such as may occur when used in a backpack.

Lock the keypad buttons

At any time, press  and  simultaneously to lock the keypad. The pump displays a **KEYS LOCKED** message and a padlock icon appears on the screen as shown.

Unlock the keypad buttons

When keypad is locked, press and release  and  simultaneously to unlock the keypad buttons. The pump displays a **KEYS UNLOCKED** message and the padlock icon disappears.

Notes:

- When keypad buttons are locked and a button is pressed, the **KEYS LOCKED** message is displayed.
-  is always active, even when the keypad is locked.
- An alarm state unlocks the keypad.
- Cycling power unlocks the keypad.


14. Advanced Settings

 **Precaution:** The following settings should only be adjusted by a qualified healthcare professional.

When the pump is off, press and hold  and only after the "HOLD FOR ON" message appears on the screen, press and hold  to access and then adjust the advanced settings.

Press  to scroll through the advanced settings. When the last setting is reached, pressing  cycles back to the first option. Use  and  to adjust a given setting. A setting is saved in memory until it is changed. Pressing  sets the on-screen value to default.

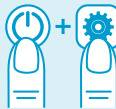Step	Button	Device Display	Function
Ensure the "HOLD FOR ON" screen is displayed before pressing and holding "SETTINGS".			
1		LOCK THERAPY? NO YES	Set the previously programmed nutritional therapy as the default and prevents any changes to this nutritional therapy by the user in normal use. Use   to set: NO: Unlocks nutritional therapy parameters (Default). YES: Locks nutritional therapy parameters. <i>Note: In Interval Mode, the Lock Therapy function is not allowed.</i>
2		ALLOW INTERVAL? YES NO	Enable / disable the use of the interval mode. Use   to set: YES: Interval Mode is available to the user. NO: Interval Mode is NOT available to the user (Default).
		COMMUNICATION OFF NURSE CALL PDMS	Enable communication for either the nurse call or the Patient Data Monitoring System (PDMS) function: Use   to set: OFF: No communication NURSE CALL: Nurse call function enabled (PDMS is disabled, even if PDMS cable is connected). PDMS: PDMS function enabled (Nurse call is disabled, even if Nurse call cable is connected). Nurse call and PDMS cables are available as accessories (see "18. Accessories"). Contact your sales representative for details.
		SET YEAR	Use   to set Year
		SET MONTH	Use   to set Month

2		SET DAY	Use   to set Day
		SET HOURS	Use   to set Hour
		SET MIN 00-59	Use   to set Minutes
		V	Show the pump software version.
		PDMS	Show the PDMS software version.
		SN	Show the pump serial number.
		LOCK THERAPY?	Return to the LOCK THERAPY screen
3			Switch off to save settings.

15. Alarms & Troubleshooting

15.1 Alarm Handling

 **Warning:** Always ensure appropriate, timely and regular patient monitoring, especially if an interruption of the tube feeding delivery may lead to a patient hazard.


When an alarm sounds:

Step	Button	Direction
1		Press the  key to acknowledge an alarm.
2		Resolve each alarm according to the specific instructions below.
3		Press the  key to resume feeding.

15.2 Alarm Overview and Troubleshooting

In case of the essential performance is lost (pump failing to deliver as intended) the following Alarm Messages could be observed:

Alarm Message	Alarm Cause	Solution
	The programmed total volume has been completely given. The target volume programmed is less than or equal to the VOLUME GIVEN THIS NUTRITIONAL THERAPY.	End administration or program a new nutritional therapy. Clear VOLUME GIVEN THIS THERAPY to start a new nutritional therapy. The reprogrammed target volume must be higher than the VOLUME GIVEN THIS THERAPY. <i>Note: If not cleared before each new nutritional therapy, the end of therapy alarm will sound prematurely.</i>
	The pump door is open or the door mechanism is damaged.	Close the pump door. If damaged, contact your sales representative or an approved service provider.
	Administration set inserted incorrectly or not at all.	Open the door, remove and reinstall the cassette and tubing. Refer to page 2 and section 6.4 of the user manual.
	Feed container and administration set are empty down to the pump. Administration set not adequately filled.	If nutritional therapy is finished, end the administration, switch pump off and flush feeding tube. If nutritional therapy is not finished: Replace or refill tube feeding container, prime the administration set and restart feeding or, Check for and release the occlusion, if any, between the container and the pump and restart feeding or, Ensure the administration set is completely filled and prime if necessary and restart feeding.  Precaution: Ensure that the administration set is not connected to a patient when priming.

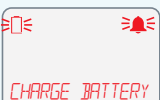	Administration set is blocked or kinked between the container and the pump.	Check for any kinking or blockage in the administration set tubing between the tube feed container and the pump. If necessary, reposition the administration set, pump or nutrition container to ensure feed can flow freely. Open the door, check the administration set installation, and re-close the door. Restart feeding.
	Administration set or feeding tube is blocked or kinked after the pump, at the patient side.	Check for any kinking or blockage in the administration set tubing between the pump and the feeding tube. If necessary, reposition the administration set or pump to ensure feed can flow freely. Check that the feeding tube is clear and not kinked. Flush feeding tube if necessary. Open the door, check the administration set installation, and re-close the door. Restart feeding.
	The battery is discharging and is running low.	
	The battery is almost fully discharged and should be plugged in immediately; otherwise Compat Ella® will shut down shortly.	Connect pump AC power via pump AC adapter (see page 4). Charge battery.
	Battery is completely depleted and must charge briefly before pump can be used. Pump will display <i>READY TO USE</i> when the pump can be restarted.	Acknowledge the <i>READY TO USE</i> message by pressing .
	The pump is in pause mode and 2 minutes have passed without any pump activity.	Resume administration or switch pump off. Extend inactivity timeout by another 2 minutes by pressing .

 REMOVE MAGNET	There is magnetic interference from an outside source.	Remove source of magnetic interference or move pump to a different location.
 ERXX RESTART PUMP	Internal equipment fault (system fault). "XX" = 01 – 99	Turn pump off and restart to clear the error. If the error returns or repeats, do not use the pump. Contact your sales representative or an approved service provider.

16. Cleaning & Disinfection

Cleaning

The pump should be cleaned on a regular basis as follows:

1. Disconnect the pump from the AC power and turn the power off before cleaning to avoid any electric shock hazard.
2. Using a damp sponge or cloth, clean the pump with warm, soapy water.
3. The pump is water resistant and may be placed under warm running water for cleaning.
4. Dry carefully with a soft cloth before reusing.

Note: To avoid damaging the pump do not use sharp objects or aggressive cleaning techniques in the pressure sensor area (see pump and accessory overview items G and H)

Disinfection

To determine proper disinfection procedure in a particular institution, follow your local institution policy. It should be noted that some harsh disinfectants may damage or discolour the pump housing which may void the warranty in this regard. Upon request, we will provide a list of approved disinfectants.

Warnings!

- Do not submerge the pump in water.
- Do not use in the dishwasher.
- To avoid electrical shock, never clean while the pump is on or plugged in.
- Do not autoclave.
- The use of harsh cleaners/disinfectants and isopropyl alcohol may damage the pump housing.

17. Maintenance

Warning: If the pump is damaged or altered, do not use it. Contact your sales representative or an approved service provider.

The Compat Ella® portable enteral feeding pump has no user-serviceable parts. No calibration is required. The Expected Service Life (ESL) is 10 years. The required maintenance interval for the Compat Ella® pump is every 5 years or sooner if the battery no longer holds a charge. In the case of a hardware or software malfunction, contact your sales representative or an approved service provider to repair or replace the pump.

18. Accessories

Patient Data Management System (PDMS) (ref: 12277770)

A separate cable is available as an accessory to enable connection of the pump to an approved PDMS system. When connected to an approved PDMS system, and PDMS is enabled, the pump will allow tracking of key information such as Rate and Volume Given via the PDMS. See advanced settings to see how to enable PDMS. Contact your sales representative for details. PDMS cable length: 2m

Nurse Call (ref: 12279521)

A separate cable is available as an accessory to enable connection of the pump to a nurse call system. When connected to an approved nurse call system, and nurse call is enabled, the pump will send an alarm signal to the nurse call management system. See advanced settings to see how to enable Nurse call. Contact your sales representative for details. Nurse Call cable length: 3m

Backpacks (ref: 12277736)

Backpacks are available as accessories. Contact your sales representative for details.

- Connect tube feeding container, administration set and pump and fully prime the system before placing into the backpack.
- Secure the pump in the backpack. Avoid excess personal items in backpack as they may cause kinks in the tubing and interfere with pump operation.
- Make sure the tube feeding container is hooked securely at the top with the set connector at the bottom.
- Make sure the administration set and tubing are free from kinks, which could cause a blockage.
- Use extra care to avoid introducing air into the system.
- Maintain backpacks in an essentially upright position to keep tube feeding container higher than pump.
- When using an open container or bag, eliminate extra air in the system as this will help avoid Air-in-Line alarm.

Table Stand / Backpack Insert (ref: 12277737)

A table stand is available as an accessory. It can also be used with a backpack. Contact your sales representative for details.

Note: some accessories might not be available locally.

19. Limited Warranty

All service should be performed by authorized personnel. Unauthorized repairs will void the warranty.

Pump, pole clamp and AC power adapter are guaranteed by the manufacturer for a period of 24 months from date of delivery.

- The warranty covers repair and replacement of components proven to be defective in material or workmanship.
- The warranty does not extend to units that have been altered or repaired by unauthorized persons and malfunctions due to improper handling and wear.
- The manufacturer can only be held responsible for the safety, reliability and performance of the feeding pump if:
 - a. Assembly, extensions, readjustments, modifications or repair have been performed by persons authorised by the manufacturer,
 - b. The electrical installation of the site where the pump is used complies with the requirements of the IEC regulations, the pump is used as specified in the Operating Instructions,
 - c. The pump is operated with the administration sets specified by the manufacturer.

20. List of Symbols

	Catalogue number		Type BF applied part		Atmospheric pressure limit (storage)
	Serial number		Class II equipment		Temperature limit (storage)
	Globe code		Keep away from rain		Battery
	Direct current		Keep away from sunlight		Fragile, handle with care
	Input voltage	IP25	Mechanical and fluid ingress		This side up
	Manufacturer		Waste of electronic and electronic equipment		Warnings and precautions
	Package content		No open flame: Fire open ignition source and smoking prohibited		CE mark and Notified Body
	Recyclable material		Not for IV use		
	Green dot		Humidity limit (storage)		
	Follow instructions for use				

21. Specifications

Mechanical

- Dimensions: L:140 mm x B:140 mm x H:62 mm.
- Weight: 499 g

Operational

- Flow Rate: 1 mL/h to 600 mL/h
- Volume: 1 mL to 4000 mL
- Increments: 1 mL
- Feed Until Empty (Continuous) CONT
- Priming Rate: ≥ 600 mL/h
- Accuracy: $\pm 5\%$ according to the following parameters:
 - Rate: 100 mL/h
 - Volume: 100 mL
 - Liquid: Water
 - Administration Set: New, previously unused set
 - Up to 1500 mL delivered volume*
- *Delivery of more than 1500 mL with one set may affect accuracy.
- Head Height above rotor: 30cm
- Head Height Effects on Accuracy: $\leq 2.5\%$ per 30cm of head height.
 - Head height range: -30cm to 60cm
- Back Pressure Effects: $\leq 0.7\%$ per kPa
- Occlusion Detection Pressure:
 - Upstream: -34 kPa for 5 seconds
 - Downstream: 83 kPa for 30 seconds
 - 103 kPa for 5 seconds

Note: Low flow rates may significantly extend the time to occlusion alarm

- Air Detection: 1.5 mL
- Operating Temperature: $+5^{\circ}\text{C}$ to $+40^{\circ}\text{C}$
- Operating Humidity: 15% to 93%, non-condensing
- Operating Atmospheric Pressure: 70 to 106 kPa

Storage

- Storage Temperature: -20°C to $+60^{\circ}\text{C}$
- Storage Humidity: 10% to 95%, non-condensing
- Atmospheric Pressure: 50 to 106 kPa

Power

- Charger:
 - Cable length: 3m
 - Input: 100-240 V; 50-60 Hz
 - Output: 5 VDC, 15.0 W
- Pump:
 - Input : 5 VDC, 10.0 W
- Fuse Type: Single blow fast acting
- Battery Type: Lithium Ion
- Battery Charge Time: ≤ 6 hours
- Battery Life: 24 hours, at 125 mL/h
- Battery capacity decay due to:
 - Storage: For optimal battery life, do not expose battery to elevated storage temperatures for extended periods. Store below 50°C when possible and do not exceed 60°C storage temperature. For extended storage, store battery at low temperature and low charge state.
 - Aging: Please note that the battery capacity numbers noted below are estimates and that actual battery capacity may vary significantly depending on factors such as actual use, charge cycles and temperature.

Pump Age	Approx. Run Time at 125 mL/h
1 yr	24 h
2 yr	24 h
3 yr	20 h
4 yr	16 h
5 yr	12 h
Above 5 years	Service Pump*

*Service interval for the Compat Ella® pump is every 5 years. Battery will be replaced. In the case of a malfunction, contact your sales representative or an approved service provider to arrange for the pump to be repaired or replaced.

- Electric shock protection: Class II
- Mechanical and Fluid ingress: IP25

22. Recycling and Disposal

Disposal of the pump:

Follow your local laws, directives, standards and guidelines for the disposal.

Disposal of the battery:

The battery should be removed as part of the disposal of the pump and processed as per specific local requirement.

Batteries contain 1,3-propane sultone.

Disposal of the packaging material:

All packaging materials have been selected according to environmentally compatible and disposal aspects and can be recycled. Please send old packaging materials to the relevant collection and reprocessing system. This way, you will contribute to the recycling or raw materials and the reduction of waste.

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These tables can be supplied translated on demand.

Guidance and manufacturer's declaration – electromagnetic immunity

The Compat Ella® pump is intended for use in the electromagnetic environment specified below.

The customer or the user of the Compat Ella® pump should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 15 kV air	± 8 kV contact ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrostatic fast transient/burst IEC 61000-4-4	± 2 kV @ PRR 100kHz for power supply lines ± 1 kV @ PRR 100kHz for input/output lines	± 2 kV @ PRR 100kHz for power supply lines ± 1 kV @ PRR 100kHz for input/output lines	AC power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line to line	± 1 kV line to line	AC power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply lines IEC 61000-4-11	70 % U_T ; 25/30 cycles single phase: at 0°	70 % U_T ; 25/30 cycles single phase: at 0°	AC power quality should be that of a typical commercial or hospital environment. The Compat Ella® pump allows continued operation during AC power interruptions via the internal battery.
	0 % U_T ; 1 cycle at 0°	0 % U_T ; 1 cycle at 0°	
	0 % U_T ; 0,5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°	0 % U_T ; 0,5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°	
	0 % U_T ; 250/300 cycle (5s)	0 % U_T ; 250/300 cycle (5s)	
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in the typical commercial or hospital environment.

NOTE U_T is the AC power voltage prior to application of the test level.

Guidance and manufacturer's declaration – electromagnetic immunity

The Compat Ella® pump is intended for use in the electromagnetic environment specified below.

The customer or the user of the Compat Ella® pump should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz outside ISM and amateur radio bands 80% AM at 1 kHz 6 Vrms 150 kHz to 80 MHz in ISM and amateur radio bands 80% AM at 1 kHz	3 Vrms 6 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the Compat Ella® pump, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d=1.2\sqrt{P}$ $d=1.2\sqrt{P}$ 80 MHz to 800 MHz $d=2.3\sqrt{P}$ 800 MHz to 6 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters [m]. Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range ^b . Interference may occur in the vicinity of equipment marked with the following symbol:
Radiated RF IEC 61000-4-3	10 V/m 80 MHz to 2.7 GHz 80% AM at 1 kHz	10 V/m	
Proximity fields from RF wireless communications equipment IEC 61000-4-3	9 V/m to 28 V/m 15 specific frequencies up to 5,785 GHz	9 V/m to 28 V/m	

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations.

Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a** Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Compat Ella® pump is used exceeds the applicable RF compliance level above, the Compat Ella® pump should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Compat Ella® pump.
- b** Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the Compat Ella® pump

The Compat Ella® pump is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Compat Ella® pump can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Compat Ella® pump as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d=1.2\sqrt{P}$	80 MHz to 800 MHz $d=1.2\sqrt{P}$	800 MHz to 6 GHz $d=2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations.

Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Guidance and manufacturer's declaration – electromagnetic emissions

The Compat Ella® pump is intended for use in the electromagnetic environment specified below. The customer or the user of the Compat Ella® pump should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The Compat Ella® pump uses RF energy on for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. The Compat Ella® pump is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

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© Unless stated otherwise, all trademarks are used with permission.