
## COMPAT® GRAVITY SETS

إرشادات الاستخدام / Návod k použití / Brugsanvisning / Gebrauchsanweisung / Instructions for use / Instrucciones de uso / Käyttöohje / Mode d'emploi / Istruzioni per l'uso / Gebruiksaanwijzing / Bruksanvisning / Instrukcja użycia / Instruções de uso / Návod na použitie / Bruksanvisning

### COMPAT® GRAVITY UNIVERSAL SET

GLOBE REF 12298230

ENPlus ENFit™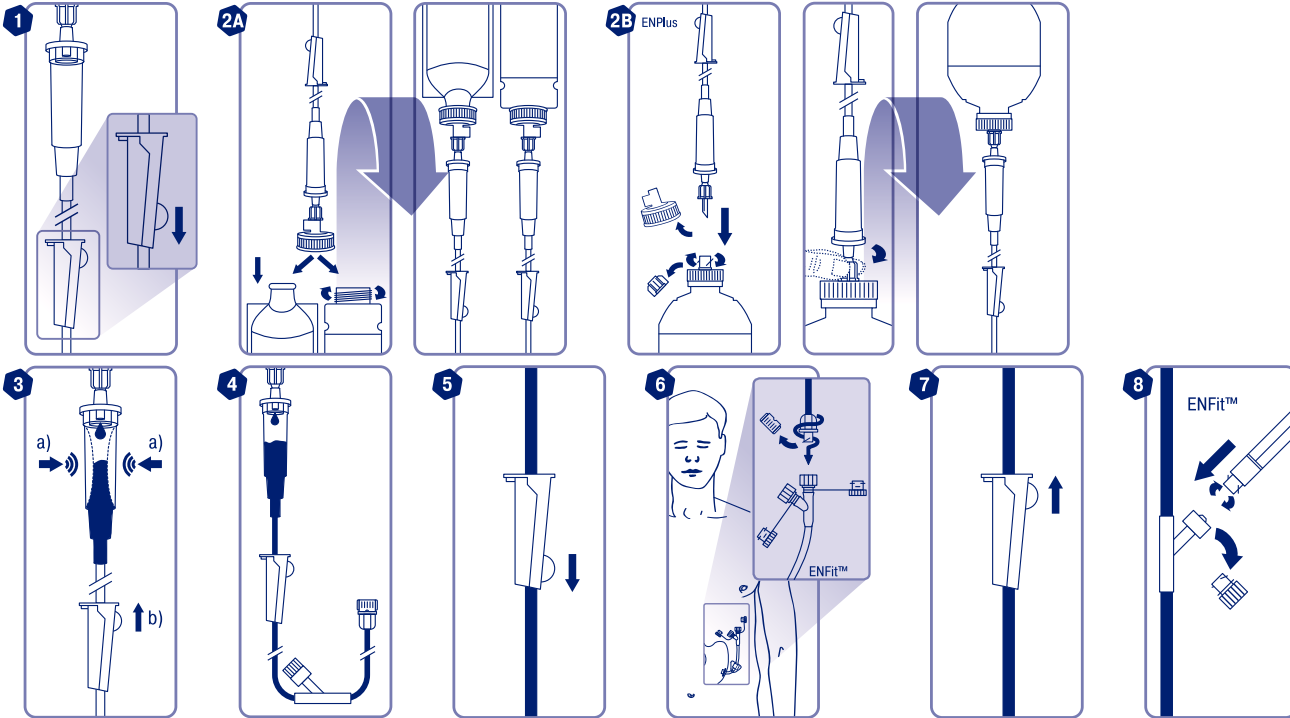


www.compat.com

### COMPAT® GRAVITY COMBISET 1.5 L

GLOBE REF 12298204

ENFit™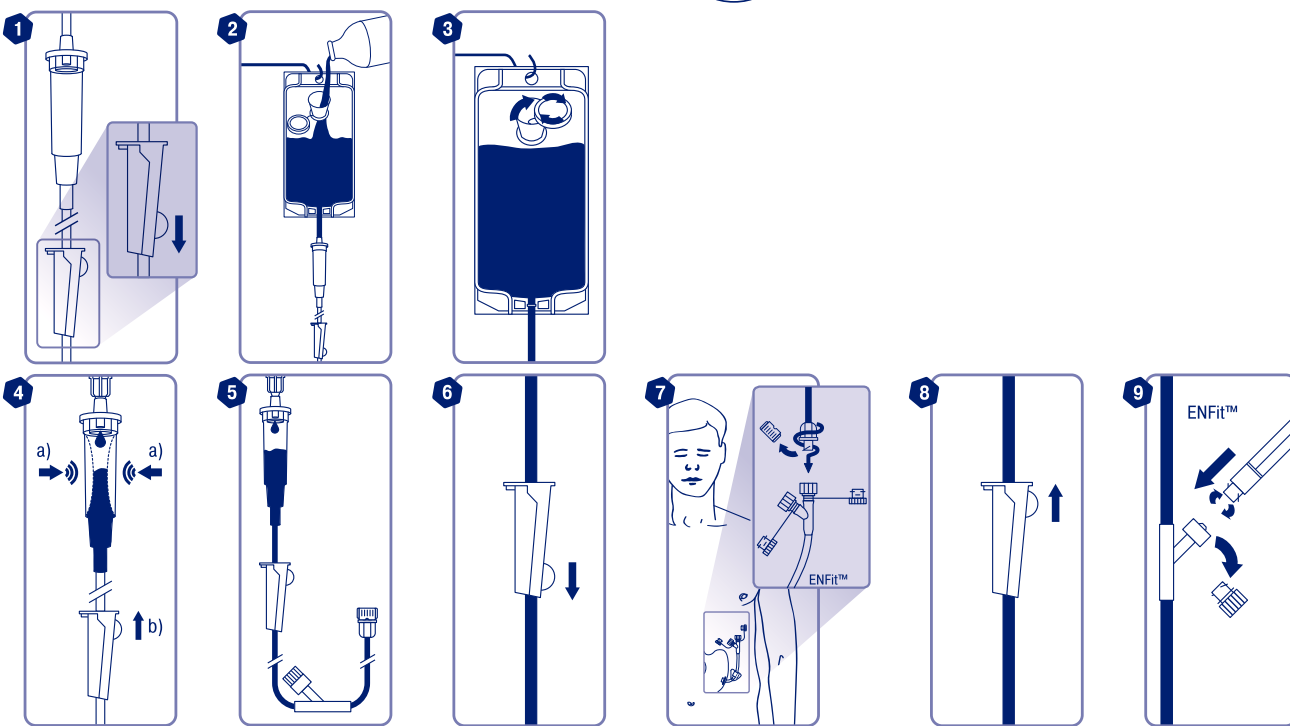


www.compat.com

### COMPAT® GRAVITY SPIKE SET

GLOBE REF 12298205

ENPlus ENFit™

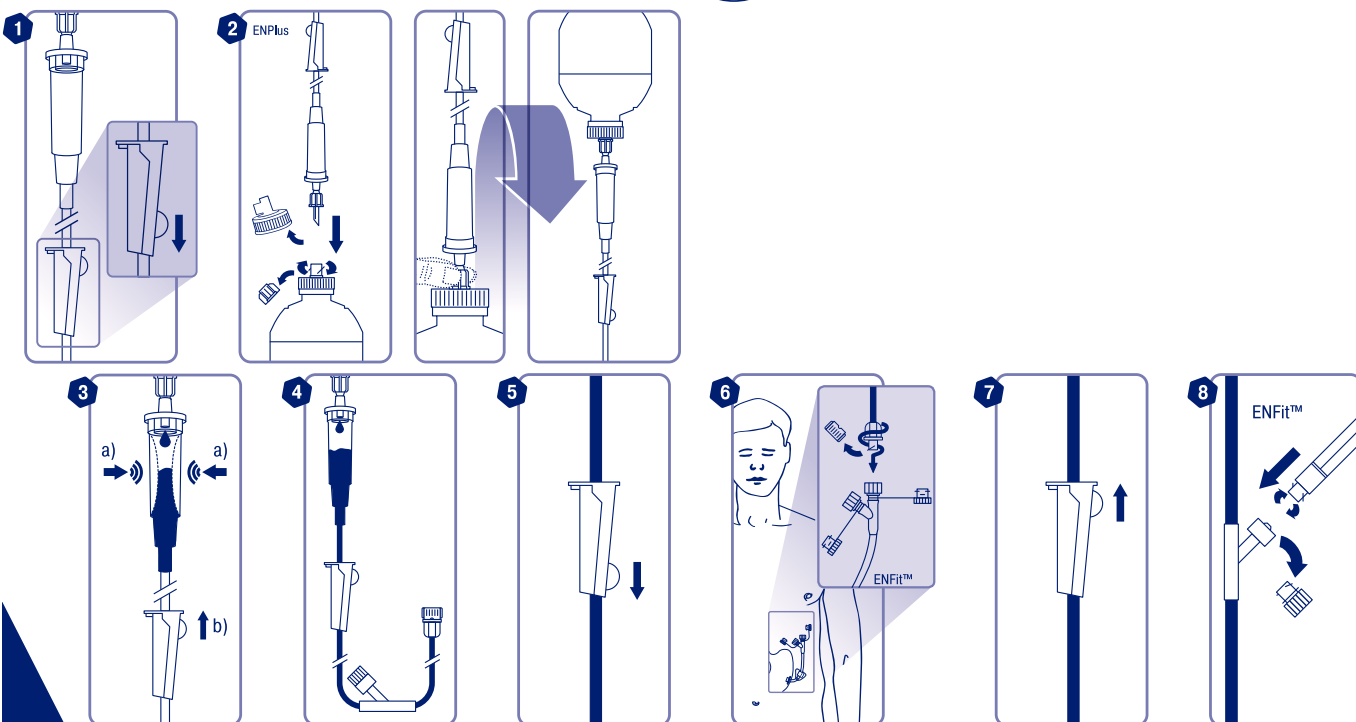www.compat.com

© Unless stated otherwise, all trademarks are owned by Société des Produits Nestlé S.A. or used with permission.

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**EN** **Enteral giving sets for tube-feeding. For gravity use.** **DEHP-free. Important Notes:** Do not use for more than 24 hours. For enteral use only. **Not** for intravenous use. Do not use with an enteral feeding pump. **Do not** use if package is damaged. Any reprocessing (especially sterilization) will compromise the structural integrity of the device. **Please Note:** The giving set may become a source of infection and have altered performance after use and in case of reuse. Follow good clinical practice and all local and national regulations for transportation and disposal. Small parts are a potential choking hazard if this device is not used as intended. Use under adult supervision. **Additional Information:** Connect the enteral giving set to the enteral feeding tube and ensure it is properly connected before feeding. Always monitor patient during feeding. Make sure the clamp is closed and the enteral giving set is disconnected from the feeding tube when not feeding.

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