

Compat[®] Modum

Closed System Gastric Residuals Aspiration and Measurement Accessory

Compat[®] Modum is intended for use as an accessory to enable closed system gastric residuals aspiration and their possible re-introduction into the stomach of the patient, when used in conjunction with appropriate tubes, syringes and collection bags. Also intended for use for enteral feeding and medication delivery.

Features at a glance

Compat[®] Modum is an accessory placed between the feeding tube and the administration set to **ease gastric fluid aspiration and GRV* measurement**.


Ease of use:

- Compat[®] Modum enables **closed system** gastric residuals aspiration into a drainage collection bag **reducing exposure to gastric fluids and their smell and lowering the risk of contamination**.
- It features two 3-way connectors which allow to **select the flow path** to easily switch from gastric emptying to enteral feeding or medication administration.
- It is **compatible** with most enteral feeding tubes, medication dispensers and drainage / suction devices**.

Safe Materials: Compat[®] Modum is **sterile, DEHP, BPA and Latex-Free*****.

Compatibility:

- Administration sets: ENFit[®]
- Enteral feeding tubes: ENFit[®] and ENLock
- Syringes: ENFit[®], ENLock and Luer
- Drainage / suction devices: Funnel**

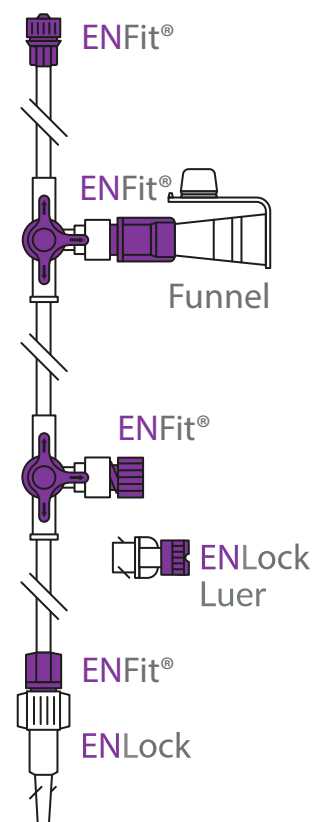


Ordering Information

Product Name	Product Code	Qty/Box
Compat [®] Modum	12340186	50

Patient Care. Our Priority.

Closed System Gastric Residuals Aspiration and Measurement Accessory

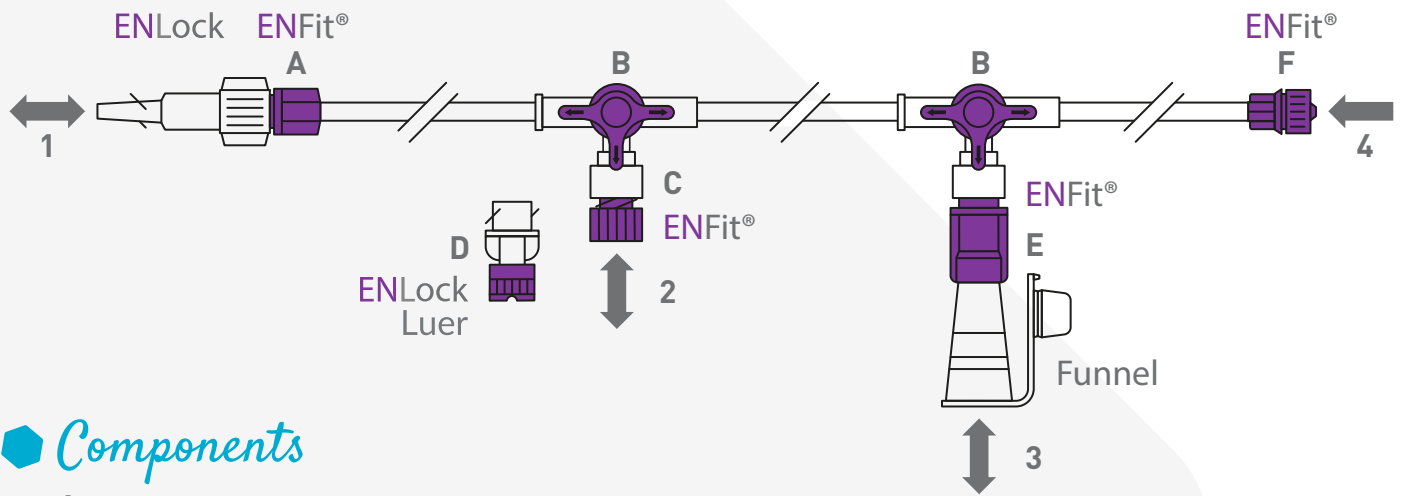

Components

- A Female ENFit® distal end with removable ENLock adapter
- B 3-way connectors to select the flow path
- C Male ENFit® drainage syringe port / medication port with cap
- D ENFit® adapter with cap compatible with Luer, Oral and ENLock medication dispensers / drainage syringes
- E Funnel connector with cap designed to be compatible with most drainage / suction devices**
- F Male ENFit® proximal connector for connection to administration set

Flow paths

- 1 To or from feeding tube (patient side connection)
- 2 To or from drainage syringe / From medication dispenser
- 3 To or from drainage/suction devices
- 4 From administration set (nutrition side connection)

Specifications

Sterile:	Sterilized with Ethylene Oxide.
Length:	12 cm.
Packing:	Individually packed in cases of 50 pieces (295x260x190mm).
Storage:	Store dry at room temperature and away from sunlight.
Shelf life:	Up to 3 years from production date. Expiry date on the package.
Duration of use:	Up to 7 days. Single Use.
ISO Standard:	ENFit® connector made according to ISO 80369-3.
CE Mark:	Complies with the European Medical Devices Directive 93/42/EEC. A CE Certification has been issued by an accredited Notified Body: TÜV SÜD (CE0123).
Legal manufacturer:	Cedic Srl, Via Liberazione 63/9, IT-20068 Peschiera Borromeo (MI).
Sustainability:	Environmental management system complies with ISO 14001:2015.
Materials:	Tubing: PVC. All materials DEHP, BPA and Latex-free***.
Country of origin:	Made in Italy.

*GRV: Gastric Residual Volume.
**Some incompatibilities may exist.

***DEHP: Not intentionally added during the manufacturing process. Not formulated with BPA or natural rubber latex.

To obtain further information please contact your Compat® representative or go to www.compat.com

®Unless stated otherwise, all trademarks are used with permission.

AXI00046 REV.2 14 JULY 2020