COMPAT® ADAPTER ENFit® TO ORAL/LUER SYRINGE

REF 12459535

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EN **Indications for use:** This transition connector is intended for connecting an enteral Oral, Luer or ENLock syringe to an enteral giving set equipped with an ENFit® medication port, in order to deliver enteral fluids. **Specifications:** Ready to use. Not made with natural rubber latex. Not formulated with DEHP or BPA. **Warnings:** Use this transition connector ONLY if an ENFit® syringe is NOT available. Use ONLY with enteral feeding systems. **Do NOT** use for intravenous use. This transition connector does not protect against inadvertent misconnection with connectors of non-enteral healthcare applications. **Do NOT** use if package is open or damaged. **Do NOT** use if labeling is incomplete or illegible. **Duration of use:** Do NOT use for more than 24 hours. **Please note:** Do not reuse, as the connector may have been contaminated during use and become a source of infection. Follow good clinical practice and national/local guidelines for safe disposal. Small parts are a potential choking hazard if this device is not used as intended. Use under adult supervision.

Instructions for Use: 1. Wash hands and use clean technique. 2. Remove protective cap (if applicable) from the enteral syringe (A). 3. Securely fit the enteral Oral (a1)/Luer slip (a2)/ENLock (a3) syringe tip into the transition connector (B) or securely screw the Luer Lock (a4) syringe tip on the transition connector (B). 4. Remove protective cap (if applicable) from the enteral medication port (C). Do not discard the protective cap. 5. Confirm the transition connector is being connected to an enteral medication port only. Do NOT connect the adapter to non-enteral connectors. 6. Remove protective cap from the transition connector. 7. Securely screw the transition connector into the ENFit® medication port. 8. Follow the instructions for use of devices connected with the transition connector. 9. When delivery by syringe is complete, in any case within a maximum of 24 hours, disconnect the transition connector together with the syringe and close the medication port with its protective cap (if applicable). 10. Discard the transition connector together with the syringe.

To obtain further information please contact your sales representative or go to www.compat.com.

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