Compat® Buddy

LOW PROFILE GASTROSTOMY FEEDING TUBE KIT

Instruction for use / Instrucciones de uso /
Bedienungsanleitung / Istruzioni per l'uso / Mode d'emploi / Návod na použitie

www.compat.com

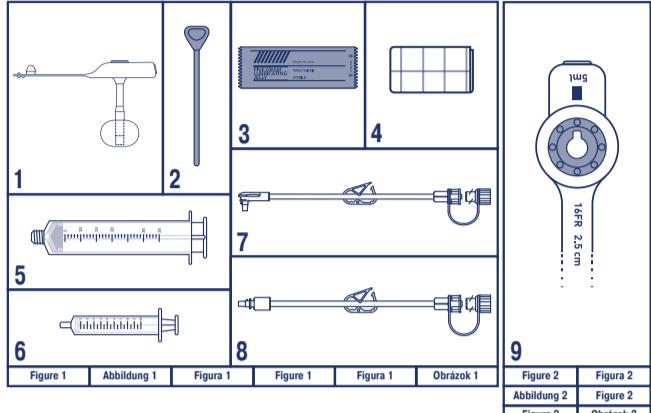
Figure 1 Abbildung 1 Figura 1 Figure 1 Figure 1 Obrázek 1

Figure 2 Abbildung 2 Figure 2 Figura 2 Obrázek 2

EN KIT CONTENT:

- 1. Compat® Buddy Gastrostomy Button (1 unit)
- 2. Stylus (1 unit)
- 3. Gel Pack sachet of aqueous gel for lubrication (1 unit)
- 4. Gauze Pad to wipe off any excess gel and clean up the stoma site (2 units)
- 5. ENFit® tip syringe 35ml, unfilled (1 unit)
- 6. Slip tip syringe 10ml, unfilled (1 unit)
- 7. Single port ENFit®, right angle connector Extension Set (1 unit)
- 8. Single port ENFit®, straight connector Extension Set (1 unit)
- 9. Purple ring

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Table 2 Tabla 2 Tabelle 2 Tabella 2 Tableau 2 Tabuľka 2

Length Longitud Länge Lunghezza Longueur Dĺžka (mm)	Size Calibre Größe Diametro Taille Velkost' (Fr/Ch)					
	12	14	16	18	20	24
8	✓	✓	✓	✓	✓	✓
10	✓	✓	✓	✓	✓	✓
12	✓	✓	✓	✓	✓	✓
15	✓	✓	✓	✓	✓	✓
17	✓	✓	✓	✓	✓	✓
20	✓	✓	✓	✓	✓	✓
23	✓	✓	✓	✓	✓	✓
25	✓	✓	✓	✓	✓	✓
27	✓	✓	✓	✓	✓	✓
30	✓	✓	✓	✓	✓	✓
35	✓	✓	✓	✓	✓	✓
40	✓	✓	✓	✓	✓	✓
45	✓	✓	✓	✓	✓	✓
50	✓	✓	✓	✓	✓	✓
55		✓		✓		
60		✓		✓		
65		✓				

CE 0123 Degania Silicone Ltd.
Degania Bet 1513000 Israel

EC REP Biometrix S.R.O.
Vicenzy 16, Samorin 93101 Slovakia

© Unless stated otherwise, all trademarks are used with permission.

86450334 REV. 0 - P102T004_00 - 28JUL2020

Compat® Buddy Gastrostomy Button (Low-Profile Gastrostomy Feeding Tube) Kit. Read all instructions carefully before use! **DESCRIPTION:** The Compat® Buddy Gastrostomy Button allows the delivery of enteral nutrition, water and medication directly to the stomach and/or gastric decompression. The available sizes are 12-24Fr. Each Fr size exists in all lengths indicated in Table 2. Product availability may vary in your country. Contact your Compat® representative for details. The size and length of each Compat® Buddy Gastrostomy Button are clearly indicated on the strap of each device and on the packaging label. **CAUTION:** The Compat® Buddy Gastrostomy Button is designed to be used in a well-established gastrostomy site. The device is provided sterile and for single use. Please inspect contents of the kit for damage. If damaged, do not use the product. **INDICATIONS FOR USE:** The Compat® Buddy Gastrostomy Button is indicated for use in patients who require long-term enteral feeding unable to tolerate oral feeding, who are at low risk for aspiration, require gastric decompression and/or medication directly into the stomach. **CONTRAINDICATIONS:** Contraindications for placement of a Compat® Buddy Gastrostomy Button include, but are not limited to, lack of adherence of the stomach to the abdominal wall, infection, lack of a well-established gastrostomy site, presence of multiple fistulas, uncertainty of the abdominal wall thickness.

WARNING:

- Please contact a healthcare professional or physician for the explanation of the warnings, care and use of the Compat® Buddy Gastrostomy Button.
- The Compat® Buddy Gastrostomy Button is for Enteral nutrition, not for IV use.
- The Compat® Buddy Gastrostomy Button is for single use only. Do not reuse, reprocess or re-sterilize. Re-use may cause a risk of infection and/or compromised functional reliability. Do not use if the package is open or damaged.
- Verify expiration date. Do not use after this date.
- Do not nick, cut, tear, puncture or otherwise damage the device.
- A single Compat® Buddy Gastrostomy Button may remain in a well-established gastrostomy for more than 30 days unless the tube is dislodged or becomes clogged. In such instances the patient should immediately contact the physician.
- The life span of the Compat® Buddy Gastrostomy Button can vary according to several factors including, but not limited to, volume of filled water in the balloon, gastric pH and tube care.
- Always keep a replacement Compat® Buddy Gastrostomy Button or a gastrostomy feeding tube available for use. Do not remove the device until a replacement Compat® Buddy Gastrostomy Button is available, to prevent the stoma from closing.
- The Extension Set is for single patient use only.
- Always follow locally applicable protocols and procedures for the use, placement, removal and care of Compat® Buddy Gastrostomy Button and accessories.

CAUTION: Placement should be performed by, or under the direction of the physician. Sizes (Fr and length) are to be subscribed by the physician.

REPLACEMENT OF THE TUBE:

- Select the Compat® Buddy Gastrostomy Button size according to the French (Fr) and length (cm) of the previous button or as subscribed by the physician. Correct size helps to prevent gastric leakage and skin irritation.
- Always wash hands before handling the device or its components.
- Inspect the device for any damage prior to use.
- Check the balloon for leakage according to the physician preferred procedures. One method consists in filling the balloon with the recommended volume of water (Table 1). The inflated balloon should be symmetrical. Do not squeeze the inflated balloon. After checking, deflate the balloon. Use Luer Slip Tip syringes only (Figure 1). **CAUTION:** Do not exceed maximum fill volume.
- Cleanse skin around the stoma.
- Insert the Stylus (Figure 1) all the way into the Feeding Port. (The use of a Stylus is optional; it provides stiffness to the shaft). Note that the Stylus should not exceed the button's distal tip.
- Lubricate the tip of the tube with water-based lubricant. Never use oil or petroleum jelly.
- Gently guide the tube through the stoma into the stomach until the entire balloon passes through the tract and the body of the Compat® Buddy Gastrostomy Button reaches the patient's skin.
- Remove the Stylus (if used in step 7).
- Hold the device and inflate the balloon with the recommended volume of water (Table 1). Use the 10ml Luer Slip tip syringe. Do not use air.
- Gently pull the Compat® Buddy Gastrostomy Button until the balloon comes up against the stomach wall. Excessive tension should be avoided as this may cause tissue necrosis. Check for any signs of gastric leakage.
- Use gauze to clean the stoma site and remove any excess lubricant.

PLACEMENT CONFIRMATION:

- Prior to the feeding tube use, confirm tube placement according to the physician preferred procedures. One method consists in aspirating gastric content. With the Extension Set connected (as instructed below) to the Compat® Buddy Gastrostomy Button, connect the ENFit® syringe to the Extension Set's Feeding Port.
- Aspirate gastric content. The presence of gastric content in the syringe confirms the correct positioning of the Compat® Buddy Gastrostomy Button position in the stomach.
- Use the ENFit® syringe to flush with 10ml of water. Check for leakage around the stoma. If there is a leak, reconfirm proper balloon inflation and verify the French size, stoma length and placement. Proper placement may be confirmed radiographically. The Compat® Buddy Gastrostomy Button has a white radiopaque stripe on the tube (Figure 3). **CAUTION:** Do not use contrast media inside the balloon.
- Use the additional gauze to clean the stoma site if required.

EXTENSION SET CONNECTION:

- Inspect Extension Sets for any damages. Do not use any damaged or clogged sets.
- If Extension Sets other than those provided in the kit are used, for instance Compat® Buddy Extension Sets, they should be used according to the manufacturer's instructions.
- Open the Feeding Port Cap.
- Insert the Extension Set to the Compat® Buddy Gastrostomy Button by aligning the connecting tooth of the Feeder (Figure 3) to the groove of the connecting ring. Align the black line on the Feeder with the black line on the Inflation Port. Press gently the feeder and turn gently clockwise ¼ turn (270 degrees) to lock the feeder in place until slight resistance is felt. **CAUTION:** Do not over-rotate past the stop point as this may damage the Connecting Tooth.
- To remove the Extension Set, gently rotate the feeder anticlockwise until the black line on the Feeder aligns with the black line on the Inflation Port, and the Connecting Tooth can be released through the groove. Remove the Extension Set and close the Feeding Port Cap.

ADMINISTRATION OF MEDICATIONS:

- Inspect Extension Set for any damages. Do not use any damaged or clogged sets.
- With the Extension Set as instructed above and the Feeding Port Cap.
- Open the Feeding Port Cap.
- Insert the Extension Set to the Compat® Buddy Gastrostomy Button by aligning the connecting tooth of the Feeder (Figure 3) to the groove of the connecting ring. Align the black line on the Feeder with the black line on the Inflation Port. Press gently the feeder and turn gently clockwise ¼ turn (270 degrees) to lock the feeder in place until slight resistance is felt. **CAUTION:** Do not over-rotate past the stop point as this may damage the Connecting Tooth.
- Use the additional gauze to clean the stoma site if required.
- Remove the Extension Set, gently rotate the feeder anticlockwise until the black line on the Feeder aligns with the black line on the Inflation Port, and the Connecting Tooth can be released through the groove. Remove the Extension Set and close the Feeding Port Cap.

EXTENSION SET CONNECTION:

- Inspect Extension Sets for any damages. Do not use any damaged or clogged sets.
- With the Extension Set as instructed above and the Feeding Port Cap.
- Open the Feeding Port Cap.
- Insert the Extension Set to the Compat® Buddy Gastrostomy Button by aligning the connecting tooth of the Feeder (Figure 3) to the groove of the connecting ring. Align the black line on the Feeder with the black line on the Inflation Port. Press gently the feeder and turn gently clockwise ¼ turn (270 degrees) to lock the feeder in place until slight resistance is felt. **CAUTION:** Do not over-rotate past the stop point as this may damage the Connecting Tooth.
- Use the additional gauze to clean the stoma site if required.
- Remove the Extension Set, gently rotate the feeder anticlockwise until the black line on the Feeder aligns with the black line on the Inflation Port, and the Connecting Tooth can be released through the groove. Remove the Extension Set and close the Feeding Port Cap.

ADMINISTRATION OF MEDICATIONS:

- Prior to the feeding tube use, confirm tube placement according to the physician preferred procedures. One method consists in aspirating gastric content. With the Extension Set connected (as instructed below) to the Compat® Buddy Gastrostomy Button, connect the ENFit® syringe to the Extension Set's Feeding Port.
- Aspirate gastric content. The presence of gastric content in the syringe confirms the correct positioning of the Compat® Buddy Gastrostomy Button position in the stomach.
- Use the ENFit® syringe to flush with 10ml of water. Check for leakage around the stoma. If there is a leak, reconfirm proper balloon inflation and verify the French size, stoma length and placement. Proper placement may be confirmed radiographically. The Compat® Buddy Gastrostomy Button has a white radiopaque stripe on the tube (Figure 3). **CAUTION:** Do not use contrast media inside the balloon.
- Use the additional gauze to clean the stoma site if required.
- Remove the Extension Set, gently rotate the feeder anticlockwise until the black line on the Feeder aligns with the black line on the Inflation Port, and the Connecting Tooth can be released through the groove. Remove the Extension Set and close the Feeding Port Cap.

ADMISSION OF MEDICATIONS:

- Prior to the feeding tube use, confirm tube placement according to the physician preferred procedures. One method consists in aspirating gastric content. With the Extension Set connected (as instructed below) to the Compat® Buddy Gastrostomy Button, connect the ENFit® syringe to the Extension Set's Feeding Port.
- Aspirate gastric content. The presence of gastric content in the syringe confirms the correct positioning of the Compat® Buddy Gastrostomy Button position in the stomach.
- Use the ENFit® syringe to flush with 10ml of water. Check for leakage around the stoma. If there is a leak, reconfirm proper balloon inflation and verify the French size, stoma length and placement. Proper placement may be confirmed radiographically. The Compat® Buddy Gastrostomy Button has a white radiopaque stripe on the tube (Figure 3). **CAUTION:** Do not use contrast media inside the balloon.
- Use the additional gauze to clean the stoma site if required.
- Remove the Extension Set, gently rotate the feeder anticlockwise until the black line on the Feeder aligns with the black line on the Inflation Port, and the Connecting Tooth can be released through the groove. Remove the Extension Set and close the Feeding Port Cap.

ADMISSION OF MEDICATIONS:

- Prior to the feeding tube use, confirm tube placement according to the physician preferred procedures. One method consists in aspirating gastric content. With the Extension Set connected (as instructed below) to the Compat® Buddy Gastrostomy Button, connect the ENFit® syringe to the Extension Set's Feeding Port.
- Aspirate gastric content. The presence of gastric content in the syringe confirms the correct positioning of the Compat® Buddy Gastrostomy Button position in the stomach.
- Use the ENFit® syringe to flush with 10ml of water. Check for leakage around the stoma. If there is a leak, reconfirm proper balloon inflation and verify the French size, stoma length and placement. Proper placement may be confirmed radiographically. The Compat® Buddy Gastrostomy Button has a white radiopaque stripe on the tube (Figure 3). **CAUTION:** Do not use contrast media inside the balloon.
- Use the additional gauze to clean the stoma site if required.
- Remove the Extension Set, gently rotate the feeder anticlockwise until the black line on the Feeder aligns with the black line on the Inflation Port, and the Connecting Tooth can be released through the groove. Remove the Extension Set and close the Feeding Port Cap.

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HK Nestlé Hong Kong Ltd.