

Compat ella®

Compat Ella® Universal Set

The Compat Ella® Universal Set is intended for the administration of enteral nutrition, fluids and/or medication and is suitable for use with the Compat Ella® enteral feeding pump.

Features at a Glance

The Compat Ella® Universal Set features a universal screw cap with air filter, an ENPlus nutrition connector, an ENFit® 3-way medication port and an ENFit® distal end (patient side tube connection). A separate bottle hanger is also included.


Reliable Feeding: The Compat Ella® Universal Set features patented fail-safe Auto-Anti-Free-Flow (AAFF) technology. AAFF is incorporated into the cassette and is always automatically activated to **ensure patient safety**.

Safer Connectors: The Compat Ella® Universal Set includes ENFit® connectors made according to ISO 80369-3 and an ENPlus connector made according to ISO 18250-3. The ENFit® and ENPlus connectors are designed to be IV incompatible to **promote patient safety** by helping to avoid accidental misconnections.

Safe Materials: The Compat Ella® Universal Set is sterile and **DEHP, BPA and Latex-Free***.

Compatibility:

- Nutrition containers: ENPlus, glass and plastic wide-neck containers, crown cork glass bottles
- Enteral feeding tubes: ENFit®
- Syringes: ENFit®

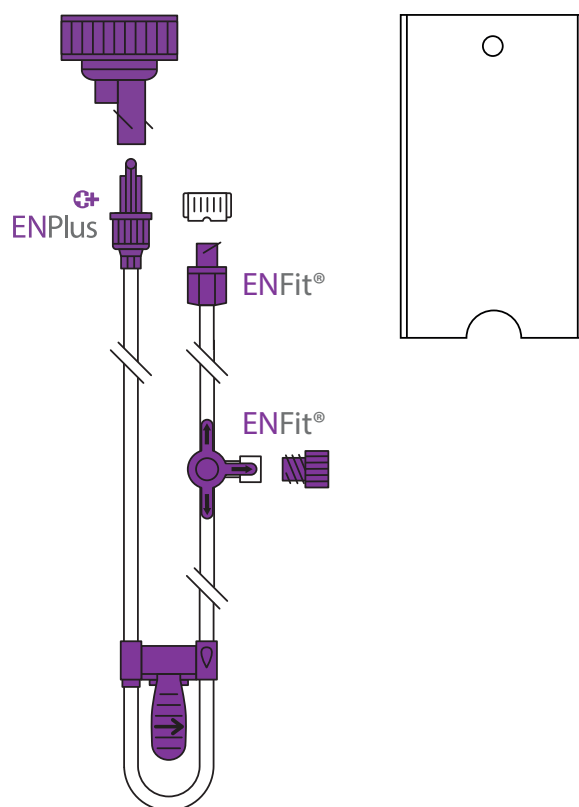


Ordering Information

Product Name	Product Code	Qty/Box
Compat Ella® Universal Set	12298206	30

Have you met Compat Ella®?

Compat Ella® Universal Set

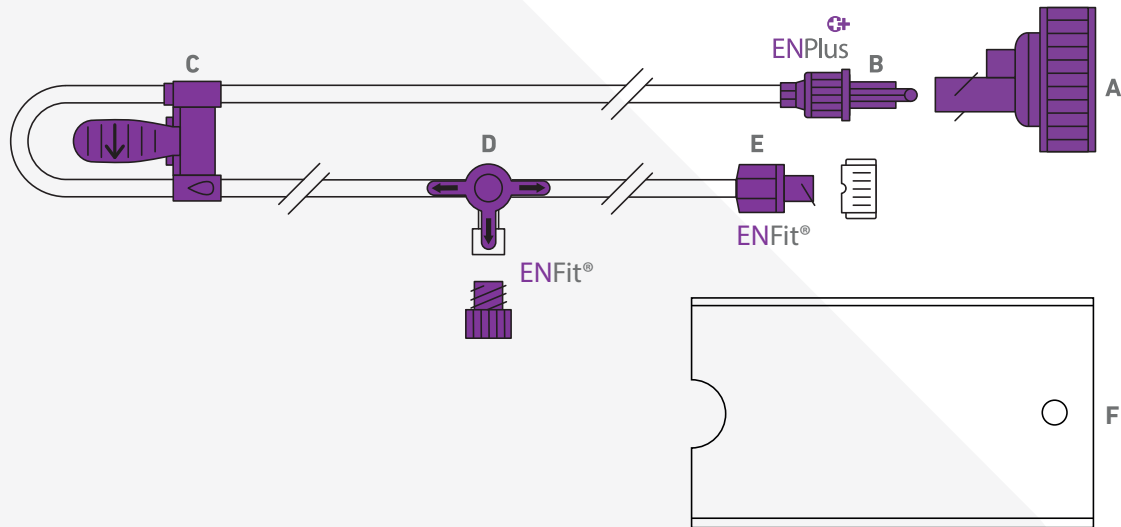

Components

- A Universal screw cap with air filter and ENPlus port compatible with glass and plastic Ø40 mm wide-neck containers (DIN 168-1) or crown cork glass bottles (EN 14634 and EN 14635)
- B ENPlus 'enteral only' connector (nutrition connection) with revolving lock
- C Compat Ella® pump cassette with integrated and patented Auto-Anti-Free-Flow (AAFF) safety feature
- D ENFit® compatible 3-way medication port with cap
- E ENFit® distal end with cap (patient side tube connection)
- F 500 ml bottle holder LDPE (Low Density Polyethylene)

Specifications

- Sterile:** Sterilized with Ethylene Oxide (EO).
- Packing:** Individually packed in cases of 30 pieces (39.5 x 29.5 x 14 cm).
- Storage:** Store at room temperature. Keep away from sunlight and humidity.
- Shelf life:** 3 years from production date. Expiry date on the package.
- Duration of use:** 24 hours. Single Use.
- Length:** Distance from nutrition connection to patient side tube connection: 230 cm.
Distance from nutrition connection to pump: 50 cm.
- Diameters:** Nominal inner diameter: 3.0 mm. Nominal outer diameter: 4.15 mm.
- ISO Standard:** ENFit® connector made according to ISO 80369-3 and ENPlus connector made according to ISO 18250-3.
- CE mark:** Complies with the European Medical Devices Directive 93/42/EEC. A CE Certification has been issued by an accredited Notified Body: TÜV SÜD (CE0123).
- Classification:** Class IIa.
- Legal manufacturer:** Cedic Srl, Via Liberazione 63/9, IT-20068 Peschiera Borromeo (MI).
- Sustainability:** Environmental Management System complies with ISO 14001:2015.
- Materials:** Tubing: PVC and silicone. All materials DEHP, BPA and Latex-free*.
Additional details on request.
- Country of origin:** Made in Italy.

*DEHP and BPA are not intentionally added during the manufacturing process. Not formulated with natural rubber latex.

To obtain further information please contact your Compat® representative or go to www.compat.com

®Unless stated otherwise, all trademarks are used with permission.

AXI00406 REV.1 08 MARCH 2021

Compat®