

Compat[®] Gravity

Compat[®] Gravity Combiset 1.5 L

The Compat[®] Gravity Combiset 1.5 L is intended for administration of enteral nutrition, fluids or medication and is suitable for gravity use.

Features at a Glance

The Compat[®] Gravity Combiset 1.5 L features an integrated 1.5 L Flexibaggle[®] reservoir, a drip chamber, an ENFit[®] Y medication port and an ENFit[®] distal end.


Patient Safety: Compat[®] Gravity Combiset 1.5 L includes ENFit[®] connectors made according to ISO 80369-3. The ENFit[®] connectors are designed to be IV incompatible to **promote patient safety** by helping to avoid accidental misconnections.

Convenience: The integrated 1.5L reservoir offers a convenient open system with a built-in hanger and screw cap. The large top-fill opening allows the bag to be refilled with nutrition or water **without stopping the feeding program, or removing the reservoir from the hanger.**

Safe Materials: The Flexibaggle[®] 1.5 L reservoir is made of Ethylene-vinyl Acetate (EVA), a soft and food compatible material. Compat[®] Gravity Combiset 1.5 L is **sterile, DEHP, BPA and Latex-Free***.

Compatibility:

- Enteral feeding tubes: ENFit[®]
- Syringes: ENFit[®]

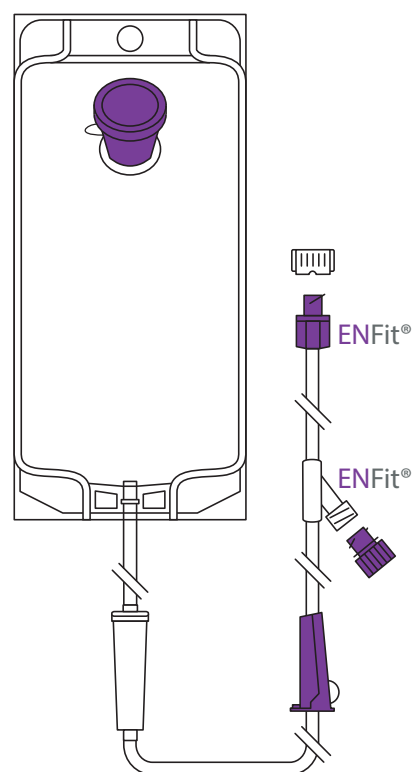


Ordering Information

Product Name	Product Code	Qty/Box
Compat [®] Gravity Combiset 1.5 L	12298204	30

Patient Care. Our Priority.

Compat® Gravity Combiset 1.5 L

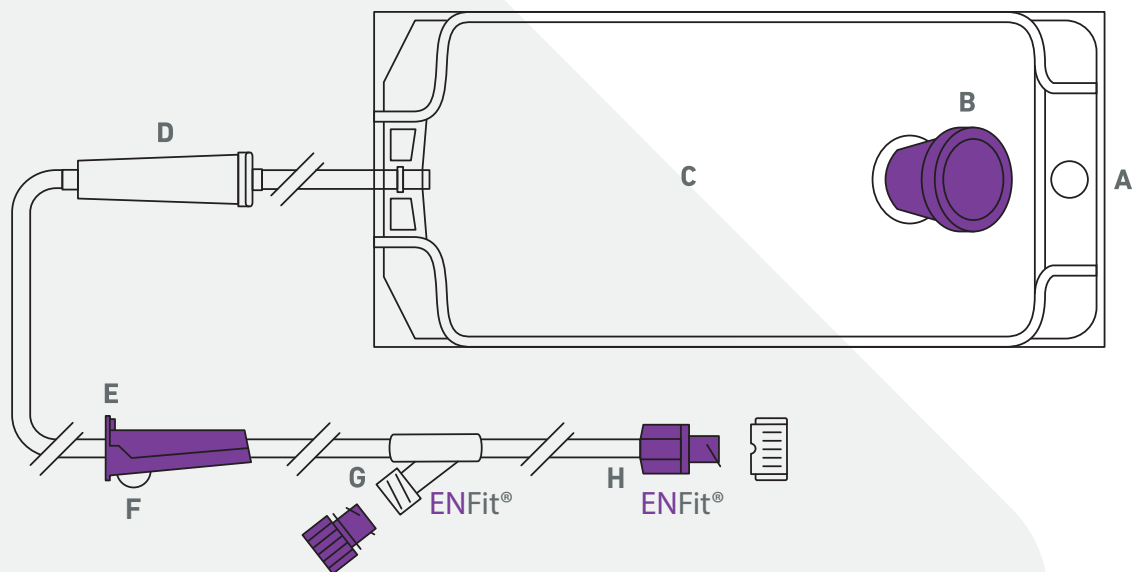

Components

- A Built in hanger
- B Large top-fill opening with screw cap
- C Flexibaggle® 1.5 L container made of EVA with graduations (at 50 mL and every 100 mL) and tags for patient name and other important information (formula, volume, date and rate)
- D Drip chamber
- E Built-in hanger to help keep the administration set distal end away from dirt
- F Roller clamp to manually shut off or adjust the flow rate
- G ENFit® Y medication port with cap
- H ENFit® distal end with cap (patient side tube connection)

Specifications

Sterile:	Sterilized with Ethylene Oxide (EO).
Packing:	Individually packed in cases of 30 pieces (58.5 x 38.5 x 19 cm).
Storage:	Store at room temperature. Keep away from sunlight and humidity.
Shelf life:	3 years from production date. Expiry date on the package.
Duration of use:	24 hours. Single Use.
Tube length:	Distance from bag to patient side tube connection: 180 cm.
Tube diameters:	Nominal inner diameter: 4.0 mm. Nominal outer diameter: 5.4 mm.
ISO standard:	ENFit® connector made according to ISO 80369-3.
CE Mark:	Complies with the European Medical Devices Directive 93/42/EEC. A CE Certification has been issued by an accredited Notified Body: TÜV SÜD (CE0123).
Classification:	Class Is.
Legal manufacturer:	Cedic Srl, Via Liberazione 63/9, IT-20068 Peschiera Borromeo (MI).
Sustainability:	Environmental management system complies with ISO 14001:2015.
Materials:	Tubing: PVC. All materials DEHP, BPA and Latex-free*. Additional details on request.
Country of origin:	Made in Italy.

*DEHP and BPA are not intentionally added during the manufacturing process. Not formulated with natural rubber latex.

To obtain further information please contact your Compat® representative or go to www.compat.com
*Unless stated otherwise, all trademarks are used with permission.

AXI00034 REV.1 01 MARCH 2021

Compat®