ES

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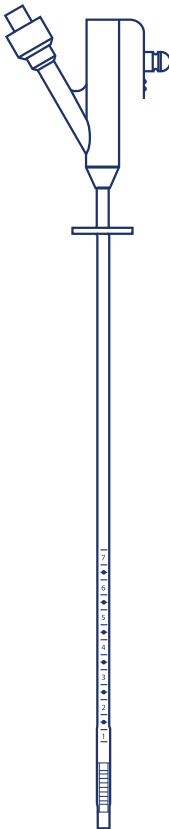
EC/REP

**Biomatrix S.R.O.**  
Vícenzy 16, Samorin 93101  
Slovakia

© Unless stated otherwise, all trademarks are used with permission.

EN

Compat® Buddy Stoma Measuring Device.

Device Description and Indication for Use:

The Compat® Buddy Stoma Measuring Device comprises a tubular shaft with graduated scale, a funnel connector with an inflation valve and a retaining balloon. The Compat® Buddy Stoma Measuring Device is made of 100% silicone. The Compat® Buddy Stoma Measuring Device is designed to determine the depth (length) of a well established gastrostomy stoma tract in order to assist in proper length selection of the Compat® Buddy Low Profile Gastrostomy tube (button). Intended for transient use (less than 60 minutes), Compat® Buddy Stoma Measuring Device size: 12 Fr 5mL. Never inflate more than 7mL.

Compat® Buddy Gastrostomy Button Kit French and Length Size(s):

| Length (mm)  | { | 8  | 10 | 12 | 15 | 17 | 20 | 23 | 25 | 27 | 30 | 35 | 40 | 45 | 50 | 55 | 60 | 65 |
|--------------|---|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|----|
| Size (Fr/Ch) | { | 12 | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  |    |    |    |
|              |   | 14 | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  |
|              |   | 16 | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  |    |    |    |
|              |   | 18 | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  |    |
|              |   | 20 | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  |    |    |    |
|              |   | 24 | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  | ✓  |    |    |    |

 Read all instructions carefully before use!

Contraindications:

Use of this device is contraindicated in individuals who do not have a well established stoma tract.

 Warning:

Each device is supplied in an individual sterile pouch. The device is guaranteed sterile unless the individual packaging has been opened or damaged. For single use only. Do not reuse, reprocess or resterilize. Re-use may cause a risk of infection and/or compromise functional reliability. Do not use if the package is open or damaged. Verify expiration date. Do not use after this date. Do not nick, cut, tear, puncture, or otherwise damage the device. Do not use the device for enteral feeding. If resistance is encountered while advancing the catheter through the stoma tract into the gastric lumen, this may indicate that the stoma has closed. Further attempts at forceful advancement of the catheter may result in damage to the abdominal wall.

 Caution:

Placement should be performed by, or under the direction of the physician. Sizes (French and length) are to be prescribed by the physician.

Placement of the Compat® Buddy Stoma Measuring Device:

1. Lubricate the tip of the measuring device with a water soluble lubricant. Do not use mineral oil or petroleum jelly (**Figure 1**).
2. Gently slide the tip of the measuring device through the stoma site into the stomach (**Figure 2**).
3. Connect a Luer Slip syringe filled with 5 mL of water to the balloon port and inflate the balloon.
4. Gently pull the device toward the abdomen until the balloon rests against the inside of the stomach wall (**Figure 3**).
5. Slide the plastic disc down until it just lightly touches the abdomen and record the measurement above the disc (**Figure 4**).
6. Take two readings, one with the patient in the upright and one in the supine position. Calculate the average of the two readings and select the next larger size.
7. Use a Luer Slip syringe to deflate the balloon before removing the Compat® Buddy Stoma Measuring Device (**Figure 5**).
8. Document the date, lot number and measured centimeter shaft length. The measurements are in 0.25 cm increments: every full centimeter is marked with a number.

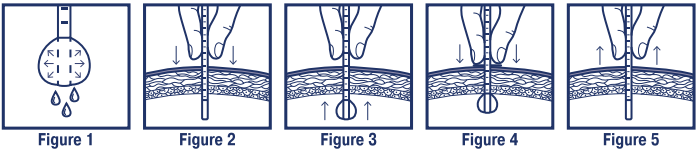
Figure 1

Figure 2

Figure 3

Figure 4

Figure 5

Warning:

Under-sizing may cause erosion embedment in the gastric wall, tissue necrosis, infection, sepsis and associated complications. Correct sizing will help to prevent gastric leakage and skin irritation.

Storage:

The products should be stored in the original packaging, at room temperature, and in a dry environment. Avoid direct sunlight and heat radiation. To obtain further information please contact your sales representative or go to [www.compat.com](http://www.compat.com).

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