Compat ella®


COMPAT ELLA® SETS

تعليمات الاستخدام / Návod k použití / Brugsanvisning / Gebrauchsanweisung / Οδηγίες χρήσης / Instructions for use / Instrucciones de uso / Käyttöohje / Mode d'emploi / Upute za uporabu / Használati utasítás / Istruzioni per l'uso / Gebruiksaanwijzing / Bruksanvisning / Instrukcja uzycia / Instruções de uso / Návod na použitie / Navodila za uporabo / Bruksanvisning / Kullanım talimatları / 使用说明

www.compat.com


COMPAT ELLA® UNIVERSAL SET

REF 12298206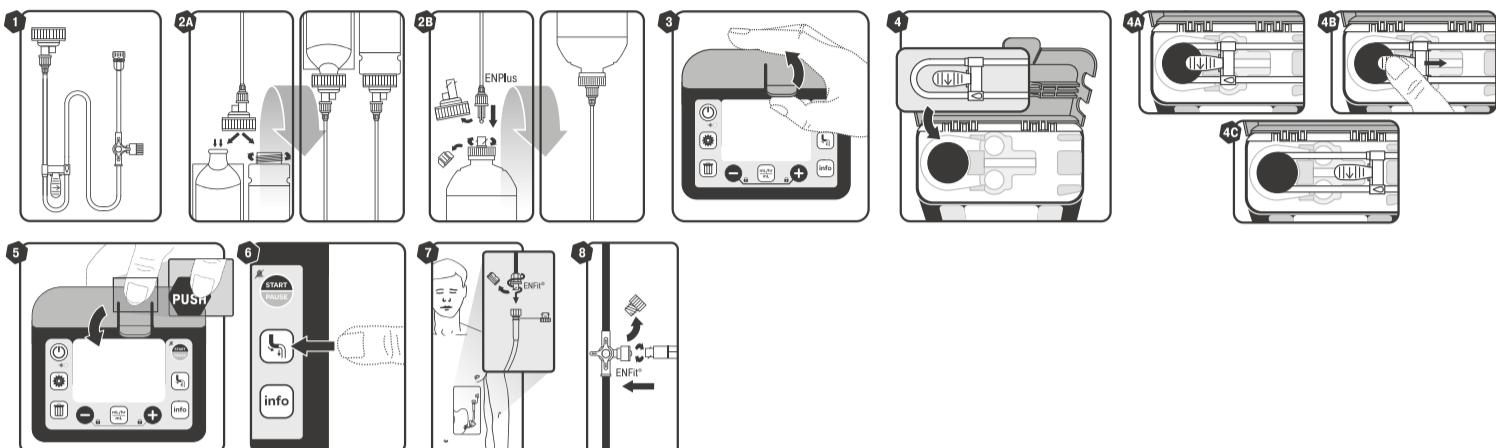


COMPAT ELLA® COMBISET 1.5L

REF 12298202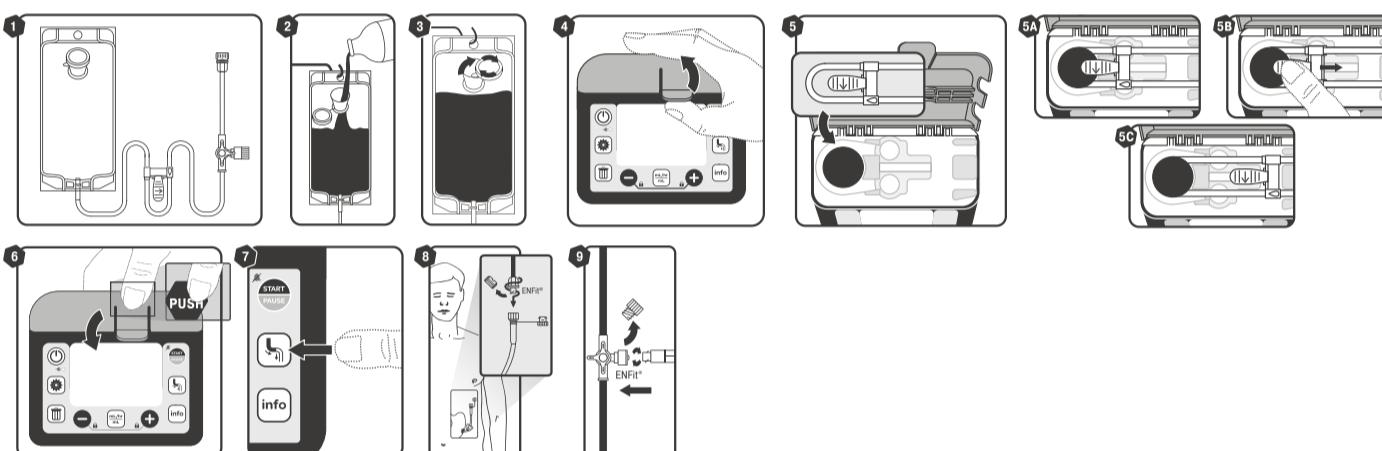


COMPAT ELLA® SPIKE SET

REF 12298203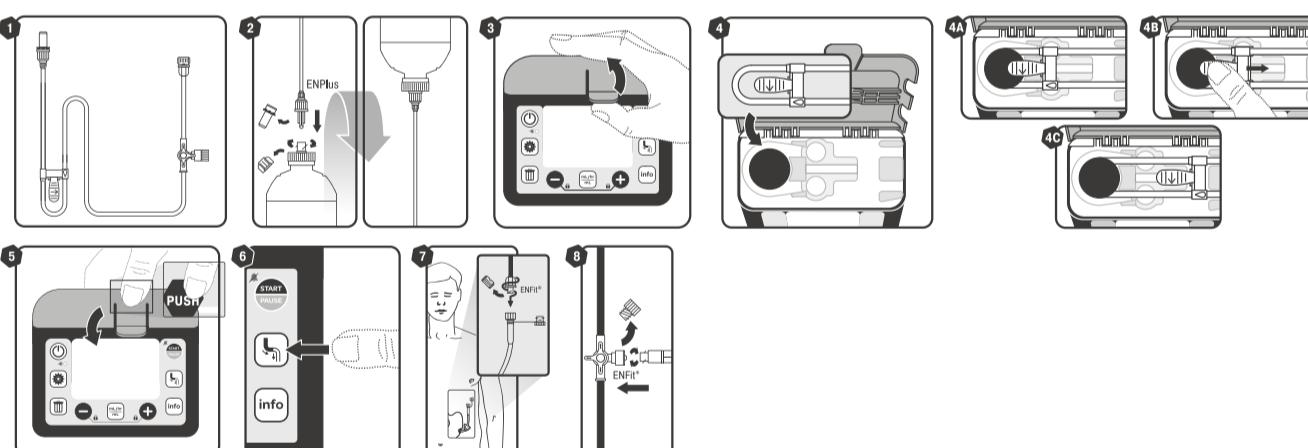

COMPAT ELLA® DUAL SPIKE SET

REF 12297288

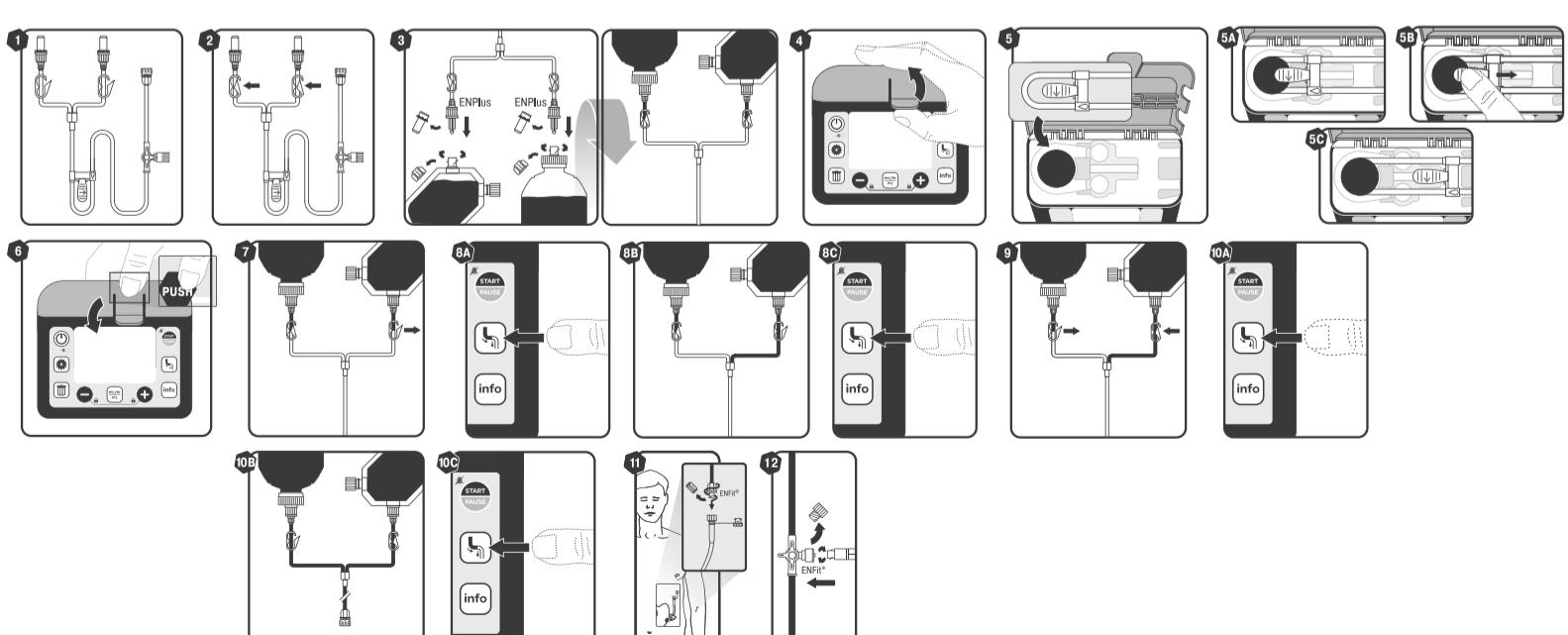STERILE EO

PHT
DEHP

N

X

2
STERILIZE

Pmax: 124 kPa
(18 psi)

CE 0123

Cedic srl

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® Unless stated otherwise, all trademarks are used with permission.

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EN **Enteral giving sets for tube-feeding. For use with Compat Ella® feeding pump.** **DEHP:** not intentionally added during the manufacturing process. **Important Notes:** Do not use for more than 24 hours. For enteral use only. **Not for intravenous use.** **Do not use if package is damaged.** Any reprocessing (especially sterilization) will compromise the structural integrity of the device. **Maximum operating pressure:** 124 kPa (18 psi). **Please Note:** The giving set may become a source of infection and have altered performance after use and in case of reuse. Follow good clinical practice and all local and national regulations for transportation and disposal. Small parts are a potential choking hazard if this device is not used as intended. Use under adult supervision. **Additional Information:** Load the enteral giving set according to the pump manual and ensure it is properly loaded before feeding. Always monitor patient during feeding. Make sure the enteral giving set is disconnected from the feeding tube when not feeding. To obtain further information please contact your sales representative or go to www.compat.com.

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