Compat ella®


COMPAT ELLA® BACKPACK

تعليمات الاستخدام / Návod k použití / Brugsanvisning / Gebrauchsanweisung / ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ / Instructions for Use / Instrucciones de uso / Käyttöohjeet / Mode d'emploi / Upute za uporabu / Használati utasítás / Istruzioni per l'uso / Naudojimo instrukcija / Gebruiksaanwijzing / Bruksanvisning / Instrukcja użycia / Instruções de utilização / Návod na použitie / Navodila za uporabo / Bruksanvisning / Kullanma Talimatı / 使用说明

www.compat.com

REF 12277736

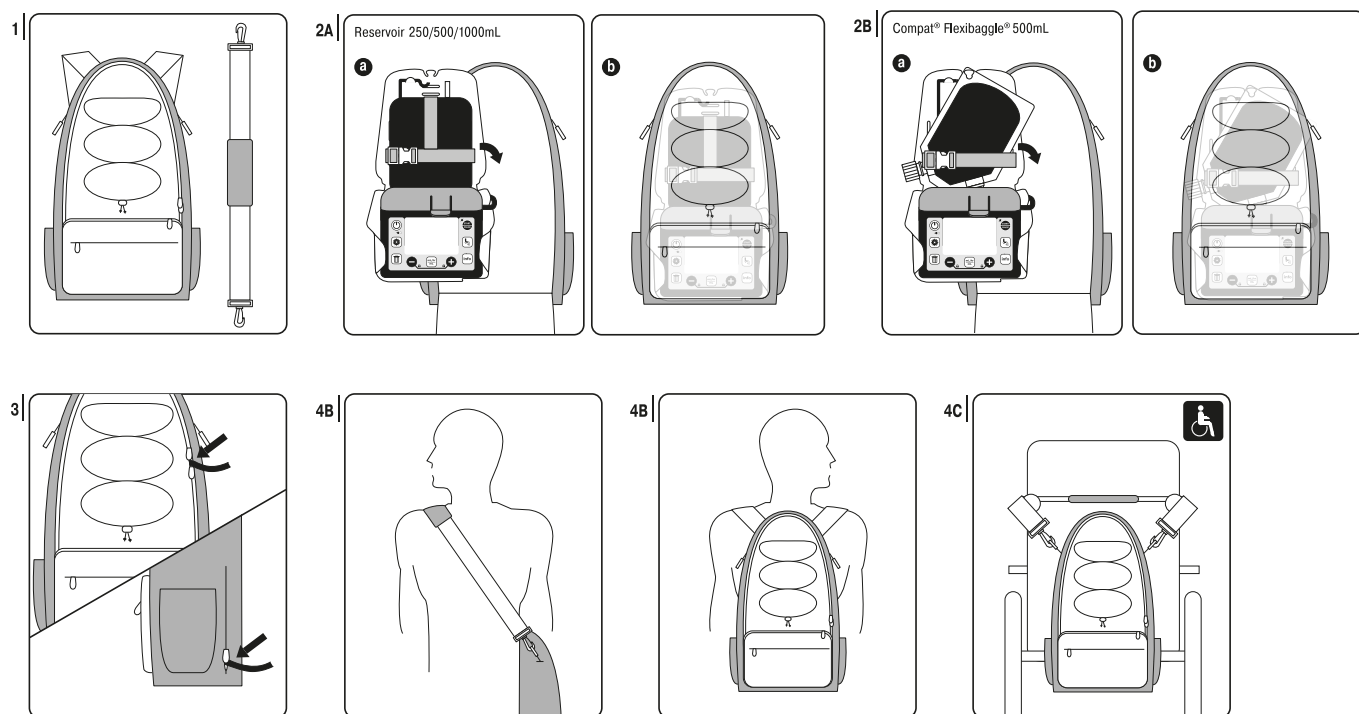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Axiom MTech S.A.
Allée de la Petite Prairie 4 - 1260 Nyon – Switzerland

Cedic srl
Via Liberazione 63/9, 20068 Peschiera Borromeo (MI), Italy

© Unless stated otherwise, all trademarks are used with permission.

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EN This accessory is intended to be used with the Compat Ella® Enteral Feeding Pump. Refer to the Compat Ella® Instructions For Use. Contact your sales representative for details.

Warnings:

Connect tube feeding container, administration set and pump and fully prime the system before placing into the backpack. Secure the pump in the backpack. Avoid excess personal items in backpack as they may cause kinks in the tubing and interfere with pump operation. Make sure the administration set and tubing are free from kinks, which could cause a blockage. When using an open container or bag, eliminate extra air in the system as this will help avoid Air-in-Line alarm.

Important notes: Ready to use. Follow good clinical practice and all local and national regulations for transportation and disposal. Immediately report serious incident to the Manufacturer and to the Competent Authority in your country.  This Instruction for Use is available at www.compat.com.

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